# All-Ceramics Preparation Guidelines Präparationsempfehlungen

Preparations as well as the images are by courtesy of Prof. Dr D Edelhoff/Germany

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## How do you achieve the perfect preparation design for all-ceramic restorations?

These guidelines on the preparation of teeth to be restored with either IPS e.max or IPS Empress all-ceramic material provide complementary information to that contained in the All-Ceramic Preparation Guide, Edition II.

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**Marginal orientation groove**

Marginale Orientierungsrolle

**Central and incisal orientation grooves**

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**Facial preparation involving 3 levels**

Faziale Präparation in  
3 Ebenen

**Initial proximal separation**

Initiale approximale  
Separation

**Final proximal preparation**

Definitive approximale  
Präparation

**Finishing and smoothing of the preparation**

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**Incisal orientation grooves**

Inzisale Orientierungsrollen

**Facial and oral aspect:  
marginal orientation groove**

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**Facial aspect: central and  
incisal orientation grooves**

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**Facial preparation involving 3 levels; oral concavity**

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**Finishing and smoothing  
of the preparation**

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# Partial crown | Teilkrone

**Pre-operative situation**

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**Cavity preparation,  
occlusal orientation grooves**

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**Occlusal and oral reduction  
of the cusp**

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**Creation of a circular  
preparation margin**

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**Proximal preparation**

Approximale Präparation

**Finished and smoothed preparation**

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**Preparation of  
the proximal box**

Approximal:  
Präparation Kasten

**Smoothing of the cavity  
and the proximal boxes**

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**Preparation after finishing  
and smoothing**

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Following cavity preparation, the shade of the prepared tooth is determined with the IPS Natural Die Material shade guide. This enables the correct materials (as regards shade, opacity and translucency) to be chosen for the all-ceramic restoration and the correct shade and brightness value to be selected.

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**Ivoclar Vivadent AG**
**Ivoclar Vivadent GmbH**