

# FRC Postec® Plus

1

Preparation of the post space | Präparation des Stiftbettes

Gates Glidden /  
Peeso





opt.  
1000-5000 rpm


Size 0: Ø max. 0.5mm

Size 1: Ø max. 0.7mm

Size 3: Ø max. 0.9mm

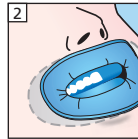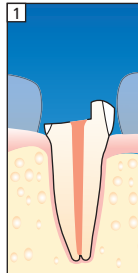

Isolation  
Absolute  
Trockenlegung


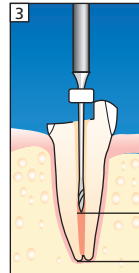


min. 4 mm


FRC Postec Plus  
Reamer



opt.  
1000-5000 rpm

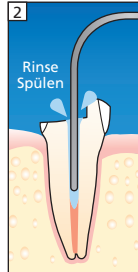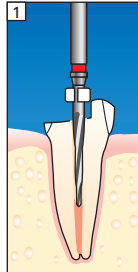


Rinse  
Spülen


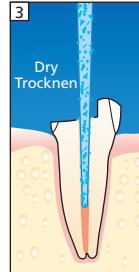


Dry  
Trocknen

FRC Postec Plus

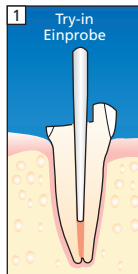


Try-in  
Einprobe

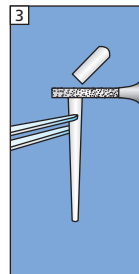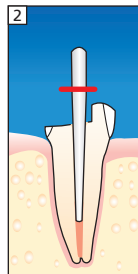


## 2 Conditioning of the post | Konditionierung des Stiftes

Total Etch

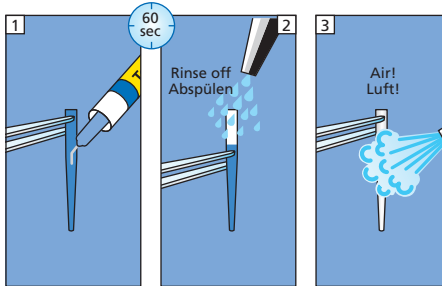

Monobond Plus

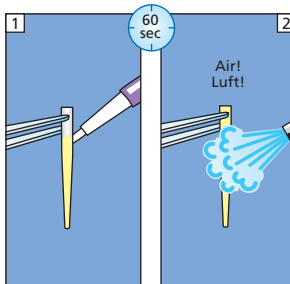

## 3 Luting of post and Core build-up | Stiftbefestigung und Stumpfaufbau

Please observe the Instructions for Use of the luting composite and the core build-up material.

Bitte beachten Sie die Gebrauchsinformation des Befestigungscomposites und des Stumpfaufbaumaterials.



Please read Instructions for Use  
Bitte lesen Sie die Gebrauchsinformation

CE 0123