# Astrobrush®

Instructions for Use  
 Gebrauchsinformation  
 Mode d'emploi  
 Istruzioni d'uso  
 Instrucciones de uso  
 Instruções de Uso  
 Bruksanvisning  
 Brugsanvisning  
 Käyttöohjeet  
 Bruksanvisning  
 Productinformatie  
 Οδηγίες Χρήσεως

Bürstchen zur Politur okklusaler und konkaver Restaurationsflächen  
 Polishing brushes for occlusal and concave restoration surfaces  
 Brossette de polissage utilisée pour les surfaces de restauration occlusales et concaves  
 Strumenti per la lucidatura delle superfici concave e occlusali del restauro  
 Capillos para el pulido de las superficies occlusales y cóncavas de restauraciones  
 Escovas de polimento para superficies occlusais e côncavas de restaurações  
 Polierborstar för restaurationer på okklusala och konkava ytor  
 Børster til polering af okklusale og konkave restaureringsflader  
 Okklusaalisten ja koverien täytepintojen kilttamisharjat  
 Børster til polering av okklusale og konkave restaureringsflater  
 Borsteljes voor het polijsten van okclusale en concave restauratievlakken  
 Βουρτσάκια στιλβωσης για μονοηλεκτρικά και κεντρικά μηχανήματα αποκαταστάσεων

CE 0123

560592/0803/2/WE3/G

## English

### Instructions for Use

#### Description:

Astrobrush are polishers for occlusal and concave restoration surfaces and for fissures in particular. The polishing effect is achieved by a polishing medium (silicon carbide), which has been incorporated into the bristles. Therefore, a polishing paste is not required.

#### Shapes:

Astrobrush is available in the following three shapes:

- Regular cup 
- Small cup 
- Point 

#### Composition:

The bristles are made of polyamide with silicon carbide incorporated as the abrasive medium. The shanks are made of gold-plated brass.

#### Indication:

- Polishing of restorations fabricated of composites, compomers, ceramics, and glass ionomer cements
- Removal of exogenous discolouration in the course of prophylactic treatment

#### Contraindication:

If patients are known to be allergic to any of the components of Astrobrush, the product should not be used.

#### Side effects:

None known to date.

#### Application:

##### Regular cup:

For polishing restorations fabricated of composites, compomers, resin-reinforced glass ionomer cements, and ceramics. This polisher is particularly suitable for large restoration surfaces in the posterior region. It also represents the basic instrument for the removal of exogenous discolouration in the course of prophylactic treatment.

##### Small cup:

For polishing anterior and cervical restorations and the occlusal surfaces of tooth-coloured posterior restorations. Removal of discolouration near the gingiva in the course of prophylactic treatment.

##### Point:

For polishing deep, occlusal fissures in tooth-coloured posterior restorations. Removal of exogenous discolouration in proximal areas in the course of prophylactic treatment.

#### Notes:

- Recommended speed: 5000 rpm
- Use only slight pressure
- Do not use a water spray
- Avoid heat build-up
- Astrobrush is autoclavable at up to 140 °C (284 °F)
- Polishing paste is not required

#### Warning:

Do not inhale polishing dust.

#### Store out of the reach of children.

For dental use only.

#### Date information prepared:

08/2003

#### Ivoclar Vivadent AG

9494 Schaan/Liechtenstein

This product has been developed solely for use in dentistry and should be used strictly according to the Instructions for Use. Liability cannot be accepted for damages resulting from failure to observe the Instructions or the stipulated area of application. The user is responsible for testing the product for its suitability and use for any purpose not explicitly stated in the Instructions. Descriptions and data constitute no warranty of attributes and are not binding.

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**Ivoclar Vivadent S.A.**
**Ivoclar Vivadent AB**
**Ivoclar Vivadent, Inc.**