Recommended for inlays, onlays, 3/4 crowns, crowns, telescope crowns, conus crowns, posts, short and long span bridges, PFM crowns.

Au	74.8	Pt	12.6	Pd	10.2	Ag	1.0	Cu	1.3	Zn	1.0	Sn	1.0	Al	0.1	Other	0.1
----	------	----	------	----	------	----	-----	----	-----	----	-----	----	-----	----	-----	-------	-----

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Yellow, gold-based dental metal ceramic casting alloy, Type 3

Ivoclar Vivadent Worldwide

- Australia**
Ivoclar Vivadent Pty. Ltd.
1 - 5 Overseas Drive
P.O. Box 367
Noble Park, Vic. 3174
Tel. +61 3 979 595 99
Fax +61 3 979 596 45
www.ivoclarvivadent.com.au
- Brazil**
Ivoclar Vivadent Ltda.
Alameda Caiapós, 723
Centro Empresarial Tamboré
CEP 06460-110
Barueri - SP
Tel. +55 11 2424 7400
Fax +55 11 2424 7440
www.ivoclarvivadent.com.br
- Canada**
Ivoclar Vivadent Inc.
1-6600 Dixie Road
Mississauga, Ontario L5T 2Y2
Tel. +1 905 670 8499
Fax +1 905 670 3102
www.ivoclarvivadent.com
- China**
Ivoclar Vivadent (Shanghai)
Trading Co., Ltd.
2/F, Building 1,
881 Wuding Road
Jing An District
Shanghai 200040
Tel. +86 21 6032 1657
Fax +86 21 6176 0988
www.ivoclarvivadent.cn
- Colombia**
Ivoclar Vivadent Marketing Ltd.
Calle 134 No. 7-8-83, Of. 520
Bogotá
Tel. +57 1 633 16 63
Fax +57 1 633 16 63
- France**
Ivoclar Vivadent SAS
B.P. 118
74410 Saint-Jorioz
Tel. +33 450 88 64 00
Fax +33 450 88 91 52
www.ivoclarvivadent.fr
- Germany**
Ivoclar Vivadent GmbH
Dr. Adolf-Schneider-Str. 2
73479 Ellwangen, Jagst
Tel. +49 79 61 8 89 0
Fax +49 79 61 63 26
www.ivoclarvivadent.de
- India**
Ivoclar Vivadent Marketing (India) Pvt. Ltd.
503/504 Raheja Plaza
15 B Shah Industrial Estate
Veera Desai Road, Andheri (West)
Mumbai, 400 053
Tel. +91 22 2673 0302
Fax +91 22 2673 0301
www.ivoclarvivadent.com.mx
- Italy**
Ivoclar Vivadent s.r.l.
Via Isonzo 67/69
40033 Casalecchio di Reno (BO)
Tel. +39 051 6113555
Fax +39 051 6113565
www.ivoclarvivadent.it
- Japan**
Ivoclar Vivadent K.K.
B.P. 118
74410 Saint-Jorioz
Tokyo 113-0033
Tel. +81 3 6903 3535
Fax +81 3 6904 3657
www.ivoclarvivadent.jp
- Mexico**
Ivoclar Vivadent S.A. de C.V.
Av. Insurgentes Sur No. 863
03810 México, D.F.
Tel. +52 55 5062 1000
Fax +52 55 5062 1029
www.ivoclarvivadent.com
- New Zealand**
503/504 Raheja Plaza
12 Omega Street, Rosedale
PO Box 303011 North Harbour
Auckland 0751
Tel. +64 9 914 9999
Fax +64 9 914 9990
www.ivoclarvivadent.co.nz
- Poland**
Ivoclar Vivadent Sp. z o.o.
ul. Jana Pawła 1/8
01175 Warszawa
Tel. +48 22 635 54 96
Fax +48 22 635 54 69
www.ivoclarvivadent.eu
- Russian Federation**
Ivoclar Vivadent Marketing Ltd.
Prospekt Andropova 18, Korpus 6
Office 10-06
115432 Moscow
Tel. +7 499 418 0300
Fax +7 499 418 0310
www.ivoclarvivadent.ru
- Spain**
Ivoclar Vivadent S.L.U.
Ctra. Fuencarral, 24-Portal I Bajó
28108 Alcobendas (Madrid)
Tel. +34 91 75 78 20
Fax +34 91 75 78 38
www.ivoclarvivadent.es
- Sweden**
Ivoclar Vivadent AB
Dahlvägen 14
169 56 Solna
Tel. +46 8 514 93 930
Fax +46 8 514 93 940
www.ivoclarvivadent.se
- Turkey**
Ivoclar Vivadent Liaison Office
Tesvikiye Mahallesi
Sakayık Sokak
Nisantasi Plaza No: 40
Kat: 6 Daire: 31-32
34021 Sisi İstanbul
Tel. +90 212 343 0802
Fax +90 212 343 0842
www.ivoclarvivadent.com.tr
- UK**
Ivoclar Vivadent Limited
Compass Building
Feldspar Close
Warrens Business Park
Enderby
Leicester LE19 4SE
Tel. +44 116 284 78 80
Fax +44 116 284 78 81
www.ivoclarvivadent.co.uk

INSTRUCTIONS FOR USE

MODELLATION
Design the framework in a reduced anatomical shape taking the planned veneer into consideration. Single crowns require a thickness of minimum 0.3 mm; abutment crowns thickness a minimum of 0.5 mm. Make sure the framework demonstrates adequate stability of shape. Avoid sharp angles. Design the connector areas to be adequate for the position and alloy being used.

SPRUNG
Provide the modeled bridge framework or coping with sprues of a suitable size. Use the direct or indirect technique being sure that the reservoir is positioned in the heat center. The connection sprues between the reservoir and the coping should be 2.5-3.0 mm in length and width.

INVESTMENT
Weigh the wax pattern including the sprue to determine the quantity of the alloy to be used. (See wax conversion sheet/formula: weight x density = gr. of alloy). Use investment following the manufacturer's instruction.

MELTING AND CASTING
Use a separate carbon/ceramic crucible for each alloy and preheat the crucible (ceramic) in the burnout furnace. Used and new alloys must be in a ratio of 1:1. Depending on the type of casting machine, follow the manufacturers instructions for use. If you use Ivoclar Vivadent Magic Wand, set propane to 0.35 bar/5 psi and oxygen to 0.7 bar/10 psi pressure. Keep the alloy in the reducing atmosphere of the flame between the inner and outer cones. Do not use flux. After casting bench cool to room temperature.

METAL PREPARATION
Carefully divest and clean the object with Al₂O₃. Do not use a hammer for divesting the object to prevent deformation. Finish the framework with carbide burs or with ceramic-bonded grinding instruments. Avoid inhalation of dust during grinding!

OXIDATION
Blast surface with 50-100 micron Al₂O₃ at max. 4.5 bar/65 psi pressure before oxidation. Subsequently, clean with ultrasonic or steam. Place the object on the ming tray and provide adequate support. Place the tray in a porcelain furnace set at a low temperature of 650C/1200F and increase the furnace temperature to 1010C/1850F without vacuum with 5 min. hold time at top temperature.

HEAT TREATMENT
Hardening: 540C/1000F for 10 minutes; air cool.
SOLDERS AND FLUXES
Design the soldering patty as small as possible and preheat it in the furnace at approximately 600C/1112F. The soldering gap should be the same thickness as the soldering strip. Allow the object to cool slowly after soldering.

POLISHING
After glazing remove oxide and flux residue and finish/polish the framework with rubber finishers/polishers.

INDICATIONS
Recommended for inlays, onlays, 3/4 crowns, crowns, telescope crowns, conus crowns, posts, short and long span bridges, PFM crowns.

CONTRAINDICATIONS
For patients with known allergy/sensitivity to any major or minor constituents of this alloy, consultation with a physician is recommended.

SIDE EFFECTS
In individual cases, sensitivity or allergies to components of this alloy may occur.

INTERACTIONS
Galvanic effects may occur between different alloys in the same oral environment.
For additional information look into the alloy property chart.

RECOMMENDATIONS
DIRECT: single crowns, inlays and onlays
INDIRECT: multiple units and multiple single crowns

INSTRUCTIONS:
1. Select a sprue with a reservoir equal to or larger than the thickest cross-section of the restoration.
2. Maintain the reservoir(s) within the heat center of the investment; keep the restoration(s) approximately 5 mm from the end of the investment and no closer than 5 mm from the sides.
3. Make sure that the sprue is connected to the thickest part of the restoration.
4. The connection between the sprue and the restoration should be flared ("trumpet" configuration); eliminating turbulence of the alloy (causing investment erosion) and facilitating an undisturbed flow of alloy during casting and solidification.
5. Maintain proper alloy weight in order to eliminate the negative effect of a too large button during solidification. The rule of thumb for proper alloy weight determination is: total wax weight x specific gravity of the alloy = proper alloy weight.

SUGGESTIONS:
1. Thickness and design of wax pattern: follow your manufacturers instructions.
2. Use chill-vents when casting heavy and/or large restorations.

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Pre-lega per saldatura: High Fusing Yellow Ceramic Solder, High Fusing Bondal Flux
Post-lega per saldatura: .615 Fine Solder, Low Fusing White Gold Solder, .585 Fine Solder, Bondal Flux
Laser weld material: Laser Ceramic Yellow

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