Au	98.6	>	Pt	8.0	>	Ag	2.5	>	Pd	1.0	>			
Cu	0.1	>	Zn	0.1	>	Sn	0.1	>	Fe	0.05	>	Other	0.1	>

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Rich yellow, gold-based dental metal ceramic casting alloy, Type 4

Golden Ceramic

Ivoclar Vivadent Worldwide

Australia Ivoclar Vivadent Pty. Ltd. 1 - 5 Overseas Drive P.O. Box 367 Noble Park, Vic. 3174 Tel. +61 3 979 595 99 Fax +61 3 979 596 45 www.ivoclarvivadent.com.au	Brazil Ivoclar Vivadent Ltda. Alameda Caiapós, 723 Centro Empresarial Tamboré CEP 06460-110 Barueri - SP Tel. +55 11 2424 7400 Fax +55 11 2424 7440 www.ivoclarvivadent.com.br	Canada Ivoclar Vivadent Inc. 1-6600 Dixie Road Mississauga, Ontario L5T 2Y2 Tel. +1 905 670 8499 Fax +1 905 670 3102 www.ivoclarvivadent.ca	China Ivoclar Vivadent (Shanghai) Trading Co., Ltd. 2/F, Building 1, 881 Wuding Road Jing An District Shanghai 200400 Tel. +86 21 6032 1657 Fax +86 21 6176 0988 www.ivoclarvivadent.cn	Colombia Ivoclar Vivadent Marketing Ltd. Calle 134 No. 7-B-83, Of. 520 Bogotá Tel. +57 1 627 33 99 Fax +57 1 633 16 63 www.ivoclarvivadent.co	France Ivoclar Vivadent SAS B.P. 118 74410 Saint-Jorioz Tel. +33 450 88 64 00 Fax +33 450 68 91 52 www.ivoclarvivadent.fr	Germany Ivoclar Vivadent GmbH Dr. Adolf-Schneider-Str. 2 73479 Ellwangen, Jagst Tel. +49 79 61 8 89 0 Fax +49 79 61 63 26 www.ivoclarvivadent.de	India Ivoclar Vivadent Marketing (India) Pvt. Ltd. 881 Wuding Road Jing An District Shanghai 200400 Tel. +86 21 6032 1657 Fax +86 21 6176 0988 www.ivoclarvivadent.cn	Indonesia Ivoclar Vivadent S.p. z.o.o. Via Isanzo 67/69 40033 Casalecchio di Reno (BO) Tel. +39 051 6113555 Fax +39 051 6113565 www.ivoclarvivadent.it	Japan Ivoclar Vivadent K.K. 1-28-24-4F Hongo Bunkyo-ku Tokyo 113-0033 Tel. +81 3 6903 3535 Fax +81 3 5844 3657 www.ivoclarvivadent.jp	Mexico Ivoclar Vivadent S.A. de C.V. Av. Insurgentes Sur No. 863 73479 Ellwangen, Jagst Tel. +52 55 5062 1000 Fax +52 55 5062 1029 www.ivoclarvivadent.com.mx	New Zealand Ivoclar Vivadent Ltd. 12 Omega Street, Rosedale PO Box 303011 North Harbour Auckland 0751 Tel. +64 9 914 9999 Fax +64 9 914 9990 www.ivoclarvivadent.co.nz	Poland Ivoclar Vivadent Sp. z o.o. ul. Jana Pawła II 78 00175 Warszawa Tel. +48 22 635 54 96 Fax +48 22 635 54 69 www.ivoclarvivadent.pl	Russian Federation Ivoclar Vivadent Marketing Ltd. Prospekt Andropova 18, Korpus 6 Office 10-06 115432 Moscow Tel. +7 499 418 0300 Fax +7 499 418 0310 www.ivoclarvivadent.ru	Spain Ivoclar Vivadent S.L.U. Ctra. Fuencarral, 24-Portal I Bajó 28108 Alcobendas (Madrid) Tel. +34 913 75 78 20 Fax +34 913 75 78 38 www.ivoclarvivadent.es	Sweden Ivoclar Vivadent AB Dvalavägen 14 169 56 Solna Tel. +46 8 514 93 930 Fax +46 8 514 93 940 www.ivoclarvivadent.se	Turkey Ivoclar Vivadent Liaison Office Tesvikeyi Mahallesi Sakayik Sokak Nisantasi Plaza No: 40 Kat: 6 Daire: 31-32 34021 Sisli Istanbul Tel. +90 212 343 0802 Fax +90 212 343 0842 www.ivoclarvivadent.com.tr	UK Ivoclar Vivadent Limited Compass Building Feldspar Close Warrens Business Park Enderby Leicester LE19 4SE Tel. +44 116 284 78 80 Fax +44 116 284 78 81 www.ivoclarvivadent.co.uk
--	---	--	--	--	--	---	---	---	---	--	---	---	---	---	--	--	--

EN INSTRUCTIONS FOR USE

MODELLATION
Design the framework in a reduced anatomical shape taking the planned veneer into consideration. Single crowns require a thickness of minimum 0.3 mm; abutment crowns thickness a minimum of 0.5 mm. Make sure the framework demonstrates adequate stability of shape. Avoid sharp angles. Design the connector areas to be adequate for the position and alloy being used.

SPRUNG
Provide the modeled bridge framework or coping with sprues of a suitable size. Use the direct or indirect technique being sure that the reservoir is positioned in the heat center. The connection sprues between the reservoir and the coping should be 2.5-3.0 mm in length and width.

INVESTMENT
Weigh the wax pattern including the sprue to determine the quantity of the alloy to be used. (See wax conversion sheet/formula: weight x density = gr. of alloy). Use investment following the manufacturer's instruction.

BURN-OUT
The suggested burnout temperature: 750-820C/1380-1510F

MELTING AND CASTING
Use a separate carbon/ceramic crucible for each alloy and preheat the crucible (ceramic) in the burnout furnace. Used and new alloys must be in a ratio of 1:1. Depending on the type of casting machine, follow the manufacturers instructions for use. If you use Ivoclar Vivadent Magic Wand, set propane to 0.35 bar/5 psi and oxygen to 0.7 bar/10 psi pressure. Keep the alloy in the reducing atmosphere of the flame between the inner and outer cones. Do not use flux. After casting bench cool to room temperature.

Casting Temperature: 1190-1250C/2180-2280F

METAL PREPARATION
Carefully divest and clean the object with Al₂O₃. Do not use a hammer for divesting the object to prevent deformation. Finish the framework with carbide burs or with ceramic-bonded grinding instruments. Avoid inhalation of dust during grinding!

OXIDATION
Blast surface with 50-100 micron Al₂O₃ at max. 4.5 bar/65 psi pressure before oxidation. Subsequently, clean with ultrasonic or steam. Place the object on the firing tray and provide adequate support. Place the tray in a porcelain furnace set at a low temperature of 650C/1200F and increase the furnace temperature to 925C/1700F without vacuum with 5 min. hold time at top temperature.

SOLDERS AND FLUXES
Design the soldering patty as small as possible and preheat it in the furnace at approximately 600C/1112F. The soldering gap should be the same thickness as the soldering strip. Allow the object to cool slowly after soldering.

Pre-Solder: HGPKF 1015 Y, High Fusing Bondal Flux, Golden Ceramic Solder
Post-Solder: .615 Fine Solder, Low Fusing White Gold Solder, .585 Fine Solder, Bondal Flux
Laser weld material: Laser Ceramic Yellow

POLISHING
After glazing remove oxide and flux residue and finish/polish the framework with rubber finishers/polishers.

INDICATIONS
Recommended for inlays, onlays, 3/4 crowns, crowns, telescope crowns, conus crowns, posts, short and long span bridges, PFM crowns.

CONTRAINDICATIONS
For patients with known allergy/sensitivity to any major or minor constituents of this alloy, consultation with a physician is recommended.

SIDE EFFECTS
In individual cases, sensitivity or allergies to components of this alloy may occur.

INTERACTIONS
Galvanic effects may occur between different alloys in the same oral environment.
For additional information look into the alloy property chart.

SPRUNG METHOD

RECOMMENDATIONS
DIRECT: single crowns, inlays and onlays
INDIRECT: multiple units and multiple single crowns

INSTRUCTIONS:
 1. Select a sprue with a reservoir equal to or larger than the thickest cross-section of the restoration.
 2. Maintain the reservoir(s) within the heat center of the investment; keep the restoration(s) approximately 5 mm from the end of the investment and no closer than 5 mm from the sides.
 3. Make sure that the sprue is connected to the thickest part of the restoration.
 4. The connection between the sprue and the restoration should be flared ("trumpet" configuration); eliminating turbulence of the alloy (causing investment erosion) and facilitating an undisturbed flow of alloy during casting and solidification.
 5. Maintain proper alloy weight in order to eliminate the negative effect of a too large button during solidification. The rule of thumb for proper alloy weight determination is: total wax weight x specific gravity of the alloy = proper alloy weight.

SUGGESTIONS:
 1. Thickness and design of wax pattern: follow your manufacturers instructions.
 2. Use chill-vents when casting heavy and/or large restorations.

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Laser weld material: Laser Ceramic Yellow

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