**Description**

Fluor Protector S is a protective fluoride varnish for tooth desensitization and caries prophylaxis.

**Composition**

Ethanol, water, polymer, additive, saccharin, mint flavouring, 1.5% ammonium fluoride (corresponds to 7700 ppm fluoride)

**Indication**

- Treatment of hypersensitive tooth necks
- Improvement of enamel resistance against acid
- Long-term caries prophylaxis
- Treatment of sensitivity after tooth whitening

**Contraindication**

If patients are known to be allergic to any of the ingredients of Fluor Protector S, the material should not be used.

**Side effects**

A slight temporary burning sensation may occur if the product comes in contact with the mucous membrane.

**Application**

Fluor Protector S is applied by dentists and dental professionals. Fluor Protector S is suitable for treating patients of all ages.

**Note:** Children under the age of seven who take systemic fluoride (fluoride tablets) should omit this treatment for several days after the application of Fluor Protector S to avoid fluoride overdose.

Generally, Fluor Protector S is applied at six-month intervals. Shorter intervals may be indicated, for example, in the event of high caries risk or erosion risk.  
 Dosing tube 4 g: The contents are sufficient for about 17 full set of teeth.  
 Dosing tube 7 g: The contents are sufficient for about 30 full set of teeth.  
 Single dose unit: The contents are sufficient for one full set of teeth.

**Step-by-step procedure**

1. Clean the tooth surfaces.
2. Create a dry working field with cotton rolls, cotton wool wads and a saliva ejector or air syringe if necessary.
3. a) Single dose units: Peel the foil top from the single dose unit; apply the varnish directly from the single dose unit.  
 b) Dispensing tube: Place the required amount in a dappen dish or similar vessel and close the tube.
4. Apply a thin layer with the help of a Vivabrush G.
5. Let the varnish dry for one minute; thereafter, remove the cotton rolls.
6. After the treatment, ask the patient to spit out the contents of the mouth without rinsing.

**Tips for patients**

Patients should refrain from eating and drinking for one hour after the treatment with Fluor Protector S.

**Storage**

- Carefully close the dispensing tube immediately after use.
- Single dose units and Vivabrush G are intended for single use only.
- The product is highly flammable and should therefore be kept away from ignition sources.
- Do not use the product after the indicated date of expiration.
- Store the product at 2–28 °C (36–82 °F).

**Keep material out of children's reach.****For use in dentistry only.**

This material has been developed solely for use in dentistry. Processing should be carried out strictly according to the Instructions for Use. Liability cannot be accepted for damage resulting from failure to observe the Instructions or the stipulated area of application. The user is responsible for testing the material for its suitability and use for any purpose not explicitly stated in the Instructions. Descriptions and data constitute no warranty of attributes and are not binding.

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[www.ivoclarvivadent.com](http://www.ivoclarvivadent.com)

**Ivoclar Vivadent Pty. Ltd.**

1 – 5 Overseas Drive, P.O. Box 367, Noble Park, Vic. 3174, Australia

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[www.ivoclarvivadent.us](http://www.ivoclarvivadent.us)

**Ivoclar Vivadent Shanghai**

Trading Co., Ltd., 2/F Building 1, 881 Wuding Road, Jing An District

200040 Shanghai, China

Tel. +86 21 6032 1657, Fax +86 21 6176 0968

[www.ivoclarvivadent.com](http://www.ivoclarvivadent.com)

**Ivoclar Vivadent Marketing Ltd.**

Calle 134 No. 7-B-83, Of. 520, Bogotá, Colombia

Tel. +57 1 627 3399, Fax +57 1 633 1663

[www.ivoclarvivadent.co](http://www.ivoclarvivadent.co)

**Ivoclar Vivadent SAS**

B.P. 118, F-74410 Saint-Jorioz, France

Tel. +33 4 50 88 64 00, Fax +33 4 50 68 91 52

[www.ivoclarvivadent.fr](http://www.ivoclarvivadent.fr)

**Ivoclar Vivadent GmbH**

Dr. Adolf-Schneider-Str. 2, D-73479 Ellwangen, Jagst, Germany

Tel. +49 7961 889 0, Fax +49 7961 6326

[www.ivoclarvivadent.de](http://www.ivoclarvivadent.de)

**Wieland Dental + Technik**

GmbH & Co. KG, Lindenstrasse 2, 75175 Pforzheim, Germany

Tel. +49 7231 3705 0, Fax +49 7231 3579 59

[www.wieland-dental.com](http://www.wieland-dental.com)

**Ivoclar Vivadent Marketing (India) Pvt. Ltd.**

503/504 Raheja Plaza, 15 B Shah Industrial Estate

Veera Desai Road, Andheri (West), Mumbai, 400 053, India

Tel. +91 22 2673 0302, Fax +91 22 2673 0301

[www.ivoclarvivadent.in](http://www.ivoclarvivadent.in)

**Ivoclar Vivadent Marketing Ltd.**

The Icon, Horizon Broadway BSD, Block M5 No. 1

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[www.ivoclarvivadent.com](http://www.ivoclarvivadent.com)

**Ivoclar Vivadent Ltd.**

12 Omega St, Rosedale, PO Box 303011 North Harbour

Auckland 0751, New Zealand

Tel. +64 9 914 9999, Fax +64 9 914 9990

[www.ivoclarvivadent.co.nz](http://www.ivoclarvivadent.co.nz)

**Ivoclar Vivadent Polska Sp. z o.o.**

ul. Jana Pawla II 78, 00-175 Warszawa, Poland

Tel. +48 22 635 5496, Fax +48 22 635 5469

[www.ivoclarvivadent.pl](http://www.ivoclarvivadent.pl)

**Ivoclar Vivadent LLC**

Prospekt Andropova 18 korp. 6/, office 10-06, 115432 Moscow, Russia

Tel. +7 499 418 0300, Fax +7 499 418 0310

[www.ivoclarvivadent.ru](http://www.ivoclarvivadent.ru)

**Ivoclar Vivadent Marketing Ltd.**

Qlaya Main St., Siricon Building No.14, 2nd Floor, Office No. 204

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[www.ivoclarvivadent.com](http://www.ivoclarvivadent.com)

**Ivoclar Vivadent Limited**

Compass Building, Feldspar Close, Warrens Business Park

Enderby, Leicester LE19 4SD, United Kingdom

Tel. +44 116 284 7880, Fax +44 116 284 7881

[www.ivoclarvivadent.co.uk](http://www.ivoclarvivadent.co.uk)

**Ivoclar Vivadent, Inc.**

175 Pineview Drive, Amherst, N.Y. 14228, USA

Tel. +1 800 533 6825, Fax +1 716 691 2285

[www.ivoclarvivadent.us](http://www.ivoclarvivadent.us)

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**Ivoclar Vivadent AB**