Indications: Onlays, 3/4 crowns, crowns, telescope crowns, conus crowns, short and long span bridges, posts.

Table with 12 columns: Au, Pt, Pd, Ag, Cu, Zn, Ni, In, Ga, Ru, Ir, Re, Other. Values range from 20.0 to 15.0.

Composition: Use a separate carbon/ceramic crucible for each alloy. Used and new alloy must be in a ratio of 1:1.

Instructions for Use: Production of wax patterns, casting, and finishing instructions in multiple languages.

Yellow, silver-based dental casting alloy, Type 4

Ivoclar Vivadent Worldwide

- List of regional offices: Australia, Brazil, Canada, China, Colombia, France, Germany, India, Italy, Japan, Mexico, New Zealand, Poland, Russia, Sweden, Switzerland, Taiwan, Turkey, USA, UK.

ISO 13485 Quality Management System Certified Made in U.S.A. www.ivoclarvivadent.com

EN INSTRUCTIONS FOR USE

MODELLATION Wax to full contour for crown and bridge frame design. For composite, build up the framework in a reduced anatomic shape taking the planned veneer into consideration.

SPRUNG Provide the modeled bridge framework or coping with sprues of a suitable size. Use the direct or indirect technique being sure that the reservoir is positioned in the heat center.

INVESTMENT Weigh the wax pattern including the sprue to determine the quantity of the alloy to be used.

BURN-OUT The suggested burnout temperature: 650-760C/1200-1400F High heat temperature investment: 650-760C/1200-1400F

MELTING AND CASTING Use a separate carbon/ceramic crucible for each alloy. Used and new alloy must be in a ratio of 1:1.

HEAT TREATMENT Annealing: 705C/1300F for 10 minutes; quench immediately (water) Hardening: 370C/700F for 30 minutes; air cool.

SOLDERS AND FLUXES Design the soldering patty as small as possible and preheat it in the furnace at approximately 600C/1112F.

POLISHING After soldering or heat treatment, remove oxide and flux residue and finish and polish the framework with rubber finishers and polishers.

INDICATIONS Recommended for onlays, 3/4 crowns, crowns, telescope crowns, conus crowns, short and long span bridges, posts.

CONTRAINDICATIONS For patients with known allergy/sensitivity to any major or minor constituents of this alloy, consultation with a physician is recommended.

SIDE EFFECTS In individual cases, sensitivity or allergies to components of this alloy may occur.

INTERACTIONS Galvanic effects may occur between different alloys in the same oral environment.

RECOMMENDATIONS DIRECT: single crowns, inlays and onlays

INDIRECT: multiple units and multiple single crowns

INSTRUCTIONS: 1. Select a sprue with a reservoir equal to or larger than the thickest cross-section of the restoration.

2. Maintain the reservoir(s) within the heat center of the investment; keep the restoration(s) approximately 5 mm from the end of the investment and no closer than 5 mm from the sides.

3. Make sure that the sprue is connected to the thickest part of the restoration.

4. The connection between the sprue and the restoration should be flared ("trumpet" configuration); eliminating turbulence of the alloy (causing investment erosion) and facilitating an undisturbed flow of alloy during casting and solidification.

5. Maintain proper alloy weight in order to eliminate the negative effect of a too large burden during solidification. The rule of thumb for proper alloy weight determination is: total wax weight x specific gravity of the alloy = proper alloy weight.

SUGGESTIONS: 1. Thickness and design of wax pattern: follow your manufacturers instructions.

2. Use chill-vents when casting heavy and/or large restorations.

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HEAT TREATMENT Annealing: 705C/1300F for 10 minutes; quench immediately (water) Hardening: 370C/700F for 30 minutes; air cool.

SOLDERS AND FLUXES Design the soldering patty as small as possible and preheat it in the furnace at approximately 600C/1112F.

POLISHING After soldering or heat treatment, remove oxide and flux residue and finish and polish the framework with rubber finishers and polishers.

INDICATIONS Recommended for onlays, 3/4 crowns, crowns, telescope crowns, conus crowns, short and long span bridges, posts.

CONTRAINDICATIONS For patients with known allergy/sensitivity to any major or minor constituents of this alloy, consultation with a physician is recommended.

SIDE EFFECTS In individual cases, sensitivity or allergies to components of this alloy may occur.

INTERACTIONS Galvanic effects may occur between different alloys in the same oral environment.

RECOMMENDATIONS DIRECT: single crowns, inlay and onlay

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