Cap Vibrator

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Seite 3

Operating Instructions

Page 12

Mode d'emploi

Page 22

Istruzioni d'uso

Pagina 33

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Pag. 42

CE

ivoclar
vivadent®
technical

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Contents

1. Views of the Unit, List of Parts	13
2. Introduction / Signs and Symbols	14
2.1 Preface	
2.2 Introduction	
2.3 Information regarding the Operating Instructions	
3. Safety First	14
3.1 Field of application	
3.2 Health and safety instructions	
4. Product Description	15
4.1 Components and functional description	
4.2 Indication / contraindication	
5. Installation and Initial Start-up	15
5.1 Unpacking and checking the contents	
5.2 Removal of transport securing screw	
5.3 Selecting the location	
5.4 Mains connection	
6. Operation	15
6.1 Switching on	
6.2 Explanation of the keys	
7. Practical Use	16
7.1 Loading the large capsule	
7.2 Mixing procedure	
7.3 Removal of large capsule	
8. Maintenance, Cleaning and Diagnosis	19
8.1 Monitoring and maintenance	
8.2 Cleaning	
8.3 Fuse replacement	
8.4 Shipment instructions for repairs and service	
9. What if...?	20
9.1 Technical malfunctions	
9.2 Repairs	
10. Product Specifications	21
10.1 Delivery form	
10.2 Technical data	

1. Views of the Unit, List of Parts

- 1 Hood
- 2 Hinge
- 3 Capsule holder
- 4 Safety rubber
- 5 Ejection lever
- 6 Capsule holder guide
- 7 Mains connection plug
- 8 Transport securing screw
- 9 Cover
- 10 Ready display
- 11 Reset button
- 12 Timer
- 13 Base

- 14 Switch (on/off)
- 15 Connector box
- 16 Fuse

2. Introduction / Signs and Symbols

2.1 Preface

Dear Customer,

Thank you for having purchased the Ivoclar Vivadent Cap Vibrator. The Cap Vibrator is an advanced technology product that is characterized by high quality and precision.

The furnace has been designed according to the latest industry standards. Inappropriate use may damage the equipment and be harmful to personnel. Please observe the relevant safety instructions and read the Operating Instructions.

We wish you much success and satisfaction with the Cap Vibrator.

2.2 Introduction

The signs and symbols in these Operating Instructions facilitate the finding of important information. They have the following meanings:

Risks and dangers

Important information

Contraindication

2.3 Information regarding the Operating Instructions

Apparatus: Cap Vibrator
Target group: Dental lab professionals

These Operating Instructions facilitate the correct, safe, and economic use of the unit.

Should you lose the Operating Instructions, extra copies can be ordered at a nominal fee from your local Service Center

3. Safety First


3.1 Field of application

This chapter is particularly important for staff who work with the apparatus or who have to carry out maintenance or repair work.

Risks and dangers

The Cap Vibrator must only be started and operated when the hood is closed.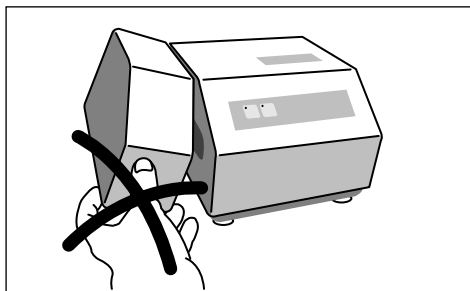

Further instructions:

- The appliance may only be used indoors.
- The appliance may only be used at altitudes of up to 4000 m above sea level
- The temperature range for faultless operation is +5 °C to +40 °C
- Maximum relative humidity of 80 % for temperatures up to 31 °C
- Voltage fluctuations should not exceed +10 % or -15 % of the voltage value
- The appliance is not suitable for use in areas where there is an explosion hazard

3.2 Health and safety instructions

The Cap Vibrator must only be used for the indications stipulated in chapter 4. Further instructions to assure the proper use of the Cap Vibrator are as follows:

- The instructions, regulations and notes in these Operating Instructions must be observed
- The unit must be properly maintained (see chapter 8)

4. Product Description

4.1 Components and functional description

The Cap Vibrator is specially designed to mix Cap products

4.2 Indication / contraindication

Indication:

- SR Ivocap
- SR Ivocap Elastomer

Contraindication:

Materials which are not designed for use with the apparatus and which are not recommended by Ivoclar Vivadent.

5. Installation and Initial Start-up

5.1 Unpacking and checking the contents

Check the delivery for completeness.

Grip the Cap Vibrator at the front and back through the cut-outs in the packaging material and carefully remove the apparatus from the styrofoam.

Contraindication


Never lift the Cap Vibrator by the hood or the capsule holder.

Check the apparatus for possible transport damage. If certain parts are damaged or missing, contact your local Customer Service.

5.2 Removal of transport securing screw

The Cap Vibrator is equipped with a transport securing screw to avoid damage during transport. Prior to the initial start-up, the transport securing screw has to be removed as follows:

Pull the Cap Vibrator to the edge of the table. Lift front slightly and remove the transport securing screw (8) from the base (13).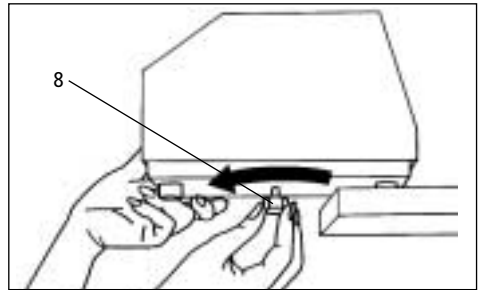

5.3 Selecting the location

Position the Cap Vibrator on level surface that is free from vibration.

5.4 Mains connection

Check to see if the voltage and frequency on the nameplate of the unit conform to the mains supply. Should this not be the case, you must not connect the unit. Please contact your local Customer Service in this case.

If the voltage conforms to the mains supply, connect to cord to the box (15) and the mains connection plug (7) to the mains.

6. Operation

6.1 Switching on

After the switch (14) has been turned on (position I), the 'ready' display will remain illuminated as long as the machine is in operation.

6.2 Explanation of the keys

Two selectable time keys are available:


- 30 seconds
- 5 minutes

Please refer to the Instructions for Use of the material for information about the mixing times.

By pushing the 'reset' button (11), the mixing procedure can be interrupted manually.


7. Practical Use

7.1 Loading the large capsule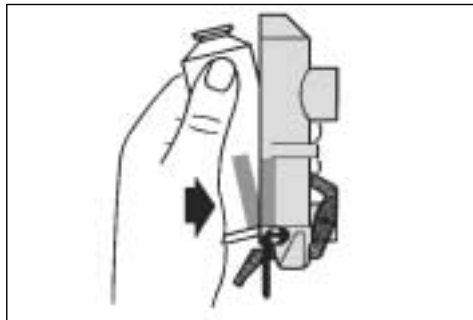


Insertion

Insert the capsule into the holder (3) with the left hand. The red mark on the capsule must correspond to the mark on the capsule holder.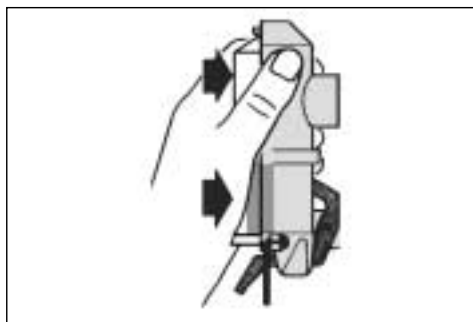

Press in

Press capsule firmly into full length of capsule holder (3). Check to see that it fits snugly!

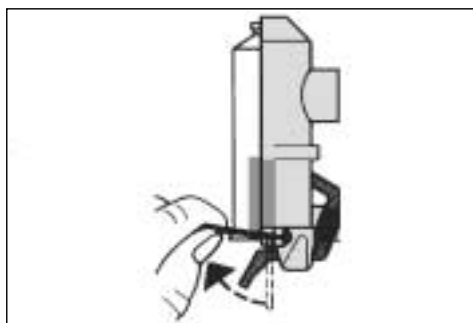Fastening

Pull rubber safety catch (4) over capsule rim. Close hood (1).

Important information

The capsule always has to be secured with the rubber safety catch (4) before the program is started.

7.2 Mixing procedure

Close hood

Always close the hood (1) before mixing. The apparatus runs only when the mixing area is closed.


Select mixing time

Pushing the desired mixing button automatically starts the mixing procedure. When the mixing time expires, the machine switches off automatically.

Interruption of procedure


Pushing the 'reset' button (11) interrupts the mixing procedure immediately. Opening the hood (1) automatically interrupts the mixing procedure.

7.3 Removal of large capsule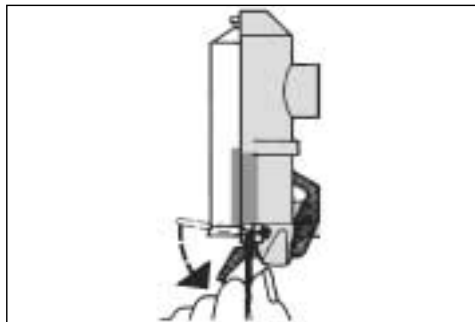


Release

Remove rubber safety catch (4) from the capsule and pull over the ejection lever (5).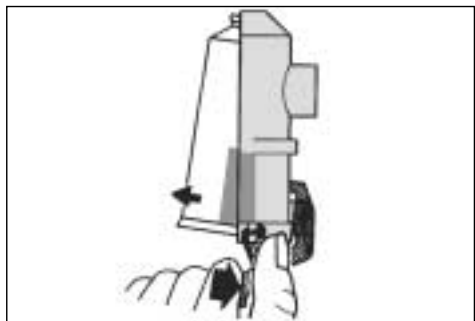

Ejection

Place left thumb in the depression provided in the holder (3) and activate ejection lever (5) with index finger.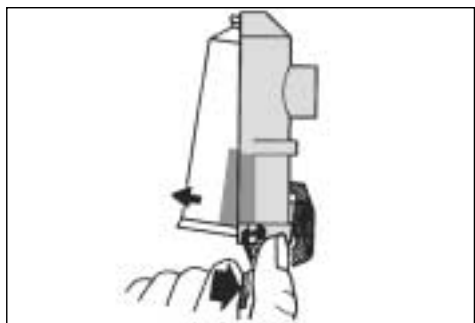

Removal

Grip ejected capsule at bottom and remove from capsule holder (3).

8. Maintenance, Cleaning and Diagnosis

This chapter describes the user maintenance and cleaning procedures that can be carried out by dental lab professionals. All other tasks must be performed by qualified service personnel at a certified Ivoclar Vivadent Service Center.

8.1 Monitoring and maintenance

The time for these maintenance procedures depends on the frequency of use and the working habits of the user. For that reason, the recommended times are only approximations.

8.2 Cleaning

Item	Frequency	Cleaning material
cover	if required	soft, dry cloth
capsule holder	if required	cleaning brush


8.3 Fuse replacement

Risks and dangers

Switch off the machine and disconnect the mains before maintenance and cleaning.

1. Open fuse holder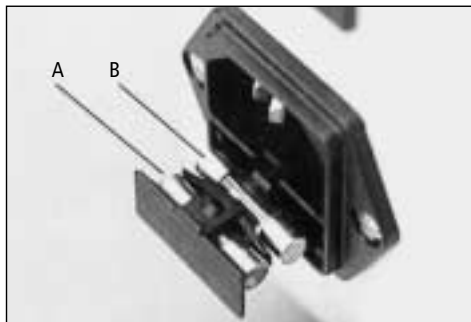

- A Main fuse
- B Reserve fuse

2. Remove defective fuse (A)
3. Remove reserve fuse (B) and put it in the forward slot.
4. Close fuse holder


Contraindications:

Do not use strong acids and solvents (e.g. MMA) that may damage the surface varnish.

8.4 Shipment for repairs and service

Fasten transport securing screw (8) to base (13) before shipping the apparatus. Do not tilt the Cap Vibrator, otherwise the screw threads cannot grip securely. Use only original packaging for shipping.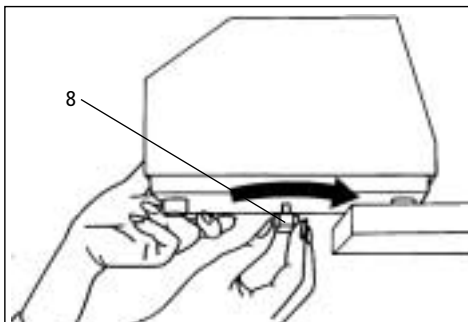

9. What if...?

This chapter will help you to recognise malfunctions and take appropriate measures, or, if possible, to perform some repairs.

9.1 Technical malfunctions

Error	Control question, cause	Measure
'Ready' display is not illuminated	Is the mains plug plugged in	Connect the mains plug to the mains supply
	Is the fuse defective	Check the fuse and replace if necessary
Capsule falls out of the capsule holder	Is the safety rubber defective or missing	Replace safety rubber

9.2 Repairs

Repairs may only be carried out by a certified Service Center. Please refer to the addresses.

If repairs during the warranty period are not carried out by a certified Service Center, the warranty will expire immediately.

Please also refer to the terms of guarantee.

10. Product Specifications

10.1 Delivery form

- 1 Cap Vibrator
- 1 Operating Instructions

Important information:
The delivery form may vary by country.

10.2 Technical data

Power supply: ac (single phase)

Standard model: 220 V/50 Hz

Special models: 240 V/50 Hz

110 V/50 Hz

110 V/60 Hz

100 V/50 Hz

100 V/60 Hz

117 V/60 Hz

Current consumption: approx. 0.68 A at 220 V

Dimensions: 275 x 235 x 285 mm
(hood raised)

Weight: 9.2 kg

Explanation of the colour designations on the fuses
according to IEC 127:

super fast blow = FF or black

quick-acting = F or red

semi-time-lag = M or yellow

time-lag = T or blue

super-time-lag = TT or grey

The Cap Vibrator is equipped with a T 3.15 A
(ø 5 x 20 mm) fuse

This apparatus has been developed solely for use in dentistry. Start-up and operation should be done strictly according to the Operating Instructions. Liability cannot be accepted for damages resulting from misuse or failure to observe the Instructions. The user is solely responsible for testing the apparatus for its suitability for any purpose not explicitly stated in the Instructions. Descriptions and data constitute no warranty of attributes and are not binding.

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Ivoclar Vivadent S.A.
Ivoclar Vivadent AB
Ivoclar Vivadent, Inc.