Brugsanvisning
Käyttöohjeet
Bruksanvisning
Productinformatie
Οδηγίες Χρήσεως
Kullanım Kilavuzu
Инструкция
Instrukcja stosowania

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For dental use only!
Caution: U.S. Federal law restricts this device to sale by or on the order of a licensed dentist.

Made in Liechtenstein
Ivoclar Vivadent AG, FL-9494 Schaan

CE 0123

5954007/210W/E3H

ivoclar
vivadent®
clinical

Instructions for Use

Product Description

Apexit Plus is an insoluble, radiopaque calcium hydroxide cement for the permanent obturation of root canals in combination with gutta-percha points. It does not shrink during setting and demonstrates excellent physical and biological properties. Apexit Plus is a two-component system. Base and activator are supplied in double-push syringes with a static mixing device.

Working Time

If no water is added, the working time is approx. 3 hours. The addition of water markedly reduces this time span.

Mixing Ratio

As Apexit Plus is supplied in double-push syringes with a static mixing device, no manual mixing is required. The material is thoroughly mixed in the correct ratio as it is dispensed and can be applied immediately.

Composition

Calcium salts (hydroxide, oxide, phosphate), hydrogenized colophony, disalicylate, bismuth salts (oxide, carbonate), highly dispersed silicon dioxide (silanized) and alkyl ester of phosphoric acid.

Indication

- Permanent obturation following vital pulp extirpation
- Permanent obturation following the removal of a gangrenous pulp and placement of intracanal disinfectant dressings
- Permanent obturation in cases with external and internal root resorption

Apexit Plus is suitable for use in the single cone and lateral condensation technique, as well as in all techniques involving heat-softened gutta-percha.

Contraindication

- Retrograde obturation
- Do not use Apexit Plus in patients with a known allergy to any of the product's ingredients.

Side Effects

Avoid contact of Apexit Plus with the skin / mucous membranes and the eyes. Unset Apexit Plus may cause slight irritations.

Interactions

The setting time of Apexit Plus in the root canal is dependent upon the availability of moisture. The setting reaction can progress very quickly in canals which have been inadequately dried. The amount of moisture required for the setting reaction reaches the root canal by means of the dentinal tubules. The material begins to set at the apex, as dentin is thinnest in this region and the apical foramen admits additional moisture. Thus even when using Apexit Plus, make sure to thoroughly dry the root canal system prior to performing the obturation. Outside the mouth, on the mixing pad, Apexit Plus may remain soft for several days, depending on the ambient humidity.

Application

Remove the cap of the double-push syringe by turning it 1/4 turn counter clock-wise (discard the cap, do not re-use it!) and replace it with a mixing tip. If the double-push syringe has been used before, replace the used mixing tip with a new one just before dispensing Apexit Plus. Insert the mixing tip. Push the tip completely down until the notch of the mixing tip is aligned with that of the double-push syringe. Secure the mixing tip in place by gripping the coloured base and turning it 1/4 turn clock-wise. The double-push syringe contains pre-dosed amounts of Apexit Plus base and activator, which are automatically mixed and dispensed when the two components are extruded. You may either pick up the mixed material directly with a lentulo spiral filler and spin it into the canal, or introduce the material into the canal with the Intra-Oral Root Canal Tips, and then use the lentulo to spin it up the canal to the apical area. Subsequently, the root canal is filled in a

conventional manner using gutta-percha points or warm gutta-percha.

At room temperature and medium humidity, Apexit Plus remains soft for several hours after mixing and dispensing. This permits obturation of multiple canals with a single mix. The setting time of Apexit Plus is between three and five hours. In very dry canals, or in situations where the sealer layer is thicker than usual (e.g. when the single cone technique is used), the setting time can be more than ten hours. Further treatment steps may be performed with any suitable material. Further manipulation of the root canal system, such as the insertion of an endodontic post or an apicectomy should not be done prior to 24 hours following root canal obturation.

Apexit Plus does not contain any pharmaceutical substances such as corticoids, antibiotics or formaldehyde-based preparations.

Thus apical foci of inflammation are not masked by the antiphlogistic and immunosuppressive effect of these additives. To avoid postoperative sensitivity in the event of an infected canal, however, antibacterial root canal dressings should be placed prior to applying Apexit Plus. Should postoperative sensitivity occur, it should cease after a maximum of 48 hours.

Storage and shelf life

- Do not use Apexit Plus after the indicated date of expiration.
- As Apexit Plus is temperature insensitive, it may either be stored at room temperature or in the refrigerator (2 to 28 °C/ 36 to 83 °F).
- Leave the used mixing tip on the syringe. Do not replace it with a new one until right before the next use. Do not re-use the original cap!
- Shelf life: see expiration date

Keep out of the reach of children!

For use in dentistry only!

Date information prepared: 12/2010; Rev. 2

Manufacturer
Ivoclar Vivadent AG
FL-9494 Schaan/Liechtenstein

This material has been developed solely for use in dentistry. Processing should be carried out strictly according to the Instructions for Use. Liability cannot be accepted for damages resulting from failure to observe the Instructions or the stipulated area of application. The user is responsible for testing the materials for their suitability and use for any purpose not explicitly stated in the Instructions. Descriptions and data constitute no warranty of attributes and are not binding.

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Ivoclar Vivadent Limited
Ivoclar Vivadent, Inc.