Instructions for Use  
Gebrauchsinformation  
Mode d'emploi  
Istruzioni d'uso  
Instrucciones de uso  
Instruções de Uso  
Bruksanvisning  
Brugsanvisning  
Käyttöohjeet  
Bruksanvisning  
Productinformatie  
Οδηγίες Χρήσεως

- Tooth Jewellery
- Zahnschmuck
- Bijou pour dent
- Gioielleria dentale
- Joyería oral
- Jóias bucais
- Tandsmykke
- Tandsmykke
- Hammasjalokivet
- Tannsmykke
- Tandversiering
- Κοσμήματα για δόντια

Made in Czech Republic  
Ivoclar Vivadent AG,  
FL-9494 Schaan/Liechtenstein

560260/0408/WE3/G

  
ivoclar  
vivadent  
clinical

### Instructions for Use

#### Description

Skyce is a range of crystal glass tooth jewellery. The stones are bonded to the labial surfaces of natural upper anterior teeth without invasive preparation.

#### Colour and sizes

Skyce is available in two sizes:

- 1.9 mm in diameter
- 2.4 mm in diameter

Colour:

- transparent

#### Step by step application

1. A rubber dam must be placed to avoid the aspiration or swallowing of the tooth jewellery.
2. The contact surface of the tooth must be professionally cleaned with brushes or cups and polishing paste, eg Proxyt.
3. A 37-% phosphoric acid gel (eg Total Etch) is applied to the contact surface. The diameter of the etched surface must be slightly larger than that of the stone.  
**Note:** The tooth must be etched for at least 60 s.
4. After 60 s of etching, thoroughly rinse off the phosphoric acid with water.

- Carefully dry the tooth surface.
- Apply a thin coat of Heliobond or Tetric Evo-Flow onto the etched region.
- Apply Heliobond or Tetric EvoFlow to the clean reverse side of Skyce and place the stone on the tooth surface. Some Heliobond or Tetric EvoFlow must come out along the sides of the stone to provide a setting for the stone.

### Recommendation

In order to optimize the bond, fully coat the stone with a thin layer of Heliobond. This does not impair the lustre of the stone.

**Note:** Tetric EvoFlow is unsuitable for this purpose.

### Important

#### Setting the tooth jewellery with Heliobond or Tetric EvoFlow.

Heliobond completely covers Skyce

Skyce with excess Heliobond

Skyce with excess Tetric EvoFlow

- The jewel must be light cured for at least 40 s both from the labial and palatal aspects (eg with bluephase®).
- If, in rare cases, excess has to be removed, only use silicone polishers. Rotary instruments must not touch the stone to prevent scratching.

### Notes

- It is easier to apply Skyce if it is held with a probe to which soft wax or a bit of adhesive material from the tip of a OpraStick has been applied.
- Do not touch the stone with your fingers or allow it to become contaminated with saliva. If Skyce is contaminated, carefully clean the stone with acrylic pellets in an alcohol bath.
- For information about the side effects and interactions of the cementation materials and the phosphoric acid gel as well as any warnings, see the corresponding Instructions for use
- Skyce can be removed whenever desired.

**Keep out of the reach of children.  
For use in dentistry only.**

Date information prepared: 04/2008  
Rev. 2

Ivoclar Vivadent AG  
FL-9494 Schaan/Liechtenstein

These materials have been developed solely for use in dentistry. Processing should be carried out strictly according to the Instructions for Use. Liability cannot be accepted for damages resulting from failure to observe the Instructions or the stipulated area of application. The user is responsible for testing the material for its suitability and use for any purpose not explicitly stated in the Instructions. Descriptions and data constitute no warranty of attributes and are not binding.

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## Ivoclar Vivadent UK Limited
## Ivoclar Vivadent, Inc.