**Description**

Variolink® Esthetic try-in pastes are water-soluble glycerine pastes. The try-in pastes enable the shade effect of all-ceramic restorations (veneers, inlays, onlays, crowns and bridges) to be simulated which are to be cemented with Variolink Esthetic luting composite.

Please note that the shade and translucency of the try-in pastes correspond to that of the **cured (!)** Variolink Esthetic luting composite.

**Shades**

The Variolink Esthetic try-in pastes are available in five shades:

Shade	Effect
Light +	Lighter / more whitish
Light	
Neutral	None
Warm	
Warm +	Darker / more yellowish

The shade of the try-in pastes slightly deviates from that of the respective uncured luting composite, as the luting composite only obtains its final shade after curing.

The shade effect of the try-in pastes is coordinated with that of the fully cured luting composite of the same thickness.

**Composition**

The Variolink Esthetic try-in pastes contain glycerine, mineral fillers and dyes.

**Indication**

Simulation of the shade of the permanent luting composite.

**Contraindication**

Do not use the try-in pastes if the patient is known to be allergic to any of the material's ingredients.

**Side effects**

The hygroscopic try-in pastes cause fluid movement within the dentin tubules. As a result, the patient may feel temporary discomfort. Therefore, the try-in procedure should be carried out under anaesthetic or after desensitization with a desensitizer (e.g. Telio® CS Desensitizer).

**Interactions**

Completely rinse any try-in paste residue from the restoration and the tooth with water prior to applying the permanent cement. Otherwise, the bond between the restoration and Variolink Esthetic may be compromised.

**Application**

- Shade simulation with the help of try-in pastes must always be carried out before the working field is isolated or the teeth are dried, since drying temporarily lightens the tooth structure.
- Apply a small amount of the selected try-in paste to the restoration and carefully seat it without pressure. If the restoration cannot be seated easily, check the inner surface of the restoration as well as the proximal regions for any premature contacts. Do not forcefully seat the restoration (risk of fracture!).

**Note:**

The occlusion of brittle ceramic restorations should be checked very cautiously or only after the restorations have been permanently cemented, as there is a risk of fracture.

- The try-in pastes are water-soluble and can be easily removed under running tap water or using ultrasonic cleaners. Make sure to thoroughly clean and dry the restoration.
- Thoroughly clean the tooth structure with water spray.
- Subsequently, adhesively cement the restoration using Variolink Esthetic (see Variolink Esthetic Instructions for Use).

**Shelf life and storage**

- Do not use Variolink Esthetic try-in pastes after the expiry date.
- Storage temperature: 2-28 °C / 36-82 °F.
- Expiry date: see note on syringes and packages.

**Keep out of the reach of children!****For use in dentistry only.**

The material has been developed solely for use in dentistry. Processing should be carried out strictly according to the Instructions for Use. Liability cannot be accepted for damages resulting from failure to observe the Instructions or the stipulated area of application. The user is responsible for testing the material for its suitability and use for any purpose not explicitly stated in the Instructions. Descriptions and data constitute no warranty of attributes and are not binding.

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# Variolink® Esthetic Try-In Paste

**EN Instructions for Use**  
– Water-soluble glycerine try-in paste

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