


Variolink® Esthetic LC


Placement of inlays (thickness < 2 mm) (glass-ceramic, e.g. IPS e.max®)

Try-in


Variolink Esthetic Try-In

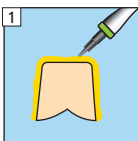


Application of Try-in paste

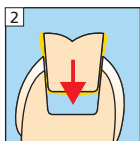


Try-in of restoration

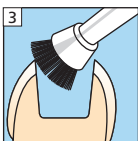



Cleaning after try-in

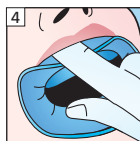



Isolation

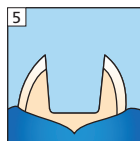


Restoration

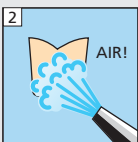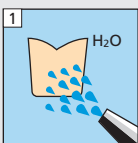



Monobond Etch & Prime

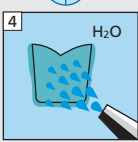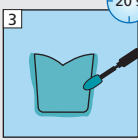


Preparation

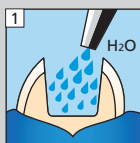

Total Etch optional

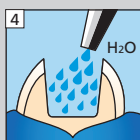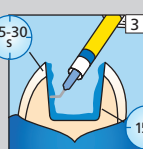


Adhese Universal

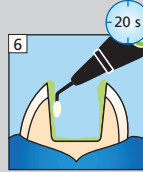

Avoid pooling





Bluephase Style

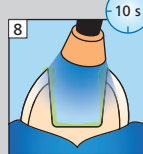


Cementation

Variolink Esthetic LC

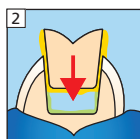
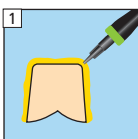


Bluephase Style

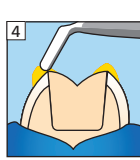


Secure the restoration until final polymerisation

Liquid Strip

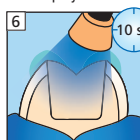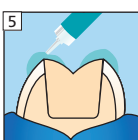


* per segment and mm ceramic

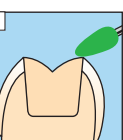

Polishing

Fluor Protector

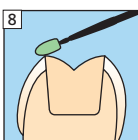

more information
www.cementation-navigation.com

CE 0123



Please read
Instruction for Use

ivoclar
vivadent
clinical


Date information prepared: 2015-07-13
681584/EN/Rev.0

Variolink® Esthetic LC


Placement of veneers (glass-ceramic, e.g. IPS e.max®)

Try-in


Variolink Esthetic Try-In

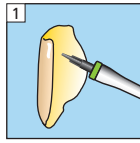


Application of Try-in paste

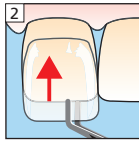


Try-in of restoration

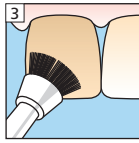





Cleaning after try-in

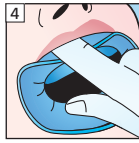

Isolation

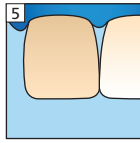


Restoration

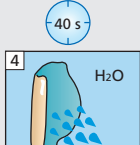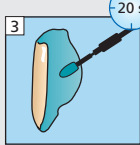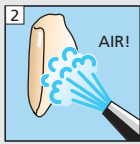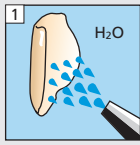



Monobond Etch & Prime





Preparation

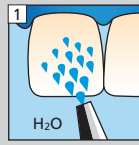


Total Etch optional

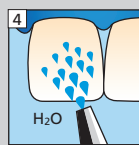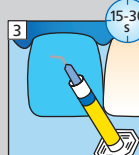

Adhese Universal

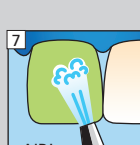
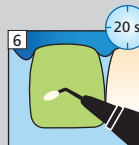


Bluephase Style

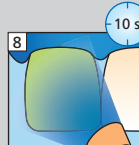


Cementation




Variolink Esthetic LC

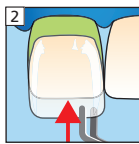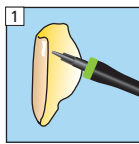


Bluephase Style

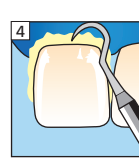

Liquid Strip

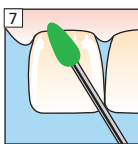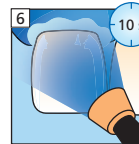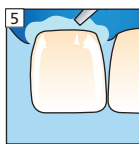

Fluor Protector

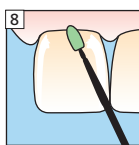

more information
www.cementation-navigation.com

CE 0123



Please read
Instruction for Use

ivoclar
vivadent
clinical