

Tetric® N-Flow Bulk Fill / Tetric® N-Ceram Bulk Fill


1

Self-Etch




or

Etch & Rinse

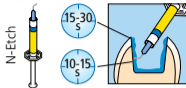


or

Selective Enamel-Etch

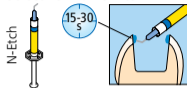

2

Tetric® N-Bond
Universal
VivaPen®

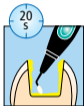

Avoid pooling



or

Tetric® N-Bond
Universal
Bottle



Avoid pooling



3

Tetric® N-Flow
Bulk Fill



Optional



4

Tetric® N-Ceram
Bulk Fill



$\geq 500 \text{ mW/cm}^2$	20 s
$\geq 1'000 \text{ mW/cm}^2$	10 s
Bluephase N®	10 s



Bluephase N®

ivoclar
vivadent
clinical

Tetric® N-Flow Bulk Fill / Tetric® N-Ceram Bulk Fill

The bulk-fill composites



	Conventional composites				Bulk Fill composites	
	FLOWABLE		SCULPTABLE		FLOWABLE NEW	SCULPTABLE
	Tetric® N-Flow		Tetric® N-Ceram		Tetric® N-Flow Bulk Fill	Tetric® N-Ceram Bulk Fill
	Dentin	Enamel	Dentin	Enamel	Dentin-like translucency	Enamel-like translucency
Layer thickness up to	1.5 mm	2 mm	1.5 mm	2 mm	4 mm	4 mm
Exposure Time $\geq 500 \text{ mW/cm}^2$	20 s					
$\geq 1000 \text{ mW/cm}^2$ or Bluephase® N (1'200 mW/cm ²)	10 s					