**Description**

Cervitec® F is a dental varnish for the mechanical protection of the tooth structure. It delivers fluoride and has antimicrobial action. The varnish is intended for the professional use in the dental practice and for the use of professionally instructed personnel under field conditions. The innovative varnish system is characterized by its good moisture tolerance during application.

**Composition**

Alcohol, Aqua, VA/Crotonates Copolymer, Cetylpyridinium chloride, Chlorhexidine diacetate, Ammonium fluoride, Aroma, Saccharin

Contains ammonium fluoride. Fluoride content: 1400 ppm

**Application**

- Caries prevention in risk groups
- Fluoridation of dental enamel
- Remineralization of incipient caries
- Protection of hyper-sensitive teeth
- Antimicrobial action

If users are known to be allergic to any of the ingredients in Cervitec F, the material should not be applied.

Cervitec F is applied by the dentist or professionally instructed personnel. Cervitec F is suitable for all age groups.

Generally, Cervitec F is applied every three months.

If required, it can be applied at shorter intervals.

Dosing tube 4 g: The contents are sufficient for about 17 full set of teeth.

Dosing tube 7 g: The contents are sufficient for about 30 full set of teeth.

Single Dose units: The contents are sufficient for 1 full set of teeth.

**Step-by-step description of the procedure**

1. Clean the tooth surfaces.
2. Isolate the application area with cotton rolls and cotton swabs. The use of a saliva ejector or air syringe is optional.
3. a) Single Dose unit: Remove the foil from the Single Dose and apply the varnish directly from the Single Dose.  
b) Dosing tube: Dispense the required amount into e.g. a dappen dish and close the tube again.
4. Apply the varnish in a thin layer using e.g. Vivabrush G.
5. Allow the varnish to dry for 1 minute and then remove the cotton rolls.
6. No rinsing after application, only spitting.

Eating and drinking should be avoided for 1 hour after application of Cervitec F.

**Notes**

Contact with the mucous membrane may result in short-term, reversible irritation.

If the varnish comes into contact with the eyes, immediately rinse with copious amounts of water and consult an ophthalmologist.

**Shelf life and storage**

- Carefully close the dosing tube immediately after use.
- The Single Dose units and the Vivabrush G are intended for single use only.
- Highly flammable. Keep away from sources of ignition.
- Do not use after the indicated date of expiration.
- Store at 2–28 °C / 36–82 °F.

**Keep out of the reach of children!****For use in dentistry only.**

The material has been developed solely for use in dentistry. Processing should be carried out strictly according to the Instructions for Use. Liability cannot be accepted for damages resulting from failure to observe the Instructions of the stipulated area of application. The user is responsible for testing the products for their suitability and use for any purpose not explicitly stated in the Instructions. Descriptions and data constitute no warranty of attributes and are not binding.

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**Ivoclar Vivadent Marketing Ltd.**
**Ivoclar Vivadent AB**
**Ivoclar Vivadent Limited**
**Ivoclar Vivadent, Inc.**