ProArt CAD Wax yellow

EN Instructions for Use

- Wax disc for the CAD/CAM technology

DE Gebrauchsinformation

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English

Properties

Chemical composition	Synthetic wax
Colour	Yellow
Dropping point	77–86 °C / 171 – 187 °F
Density	1.02 g/cm ³
Water solubility	Insoluble

Product description

ProArt CAD Wax yellow are wax discs for the CAD/CAM production of burn-out restorations in the press technique. The discs can be processed using a suitable milling machine (please observe the manufacturer's instructions).

Indications

For the fabrication of burn-out restorations in the press technique (especially suitable for IPS e.max® Press).

Contraindications

If the patient is known to be allergic to any of the components. Not approved for use directly in the patient's mouth.

Notes regarding the design

During construction it is essential to ensure of sufficient wall thickness. The minimal wall thicknesses for the press ceramic materials from Ivoclar Vivadent are stated in the Instructions for Use.

Processing with CAD/CAM

ProArt CAD Wax yellow can be processed in CAD/CAM milling systems with a standard holder of 98.5 mm. Only use milling tools recommended for this milling system when processing the material. The ambient working temperature should not exceed 35 °C / 95 °F.

Processing after milling

Finishing

After milling, separate the processed object from the disc using a suitable cross-cut tungsten carbide bur. Any finishing work or adjustments to the shape must be carried out using cross-cut tungsten carbide burs. The material can be enhanced using conventional modelling wax.

Spruing of the press channels

The rules how to choose and position the press channels correctly depend on the press ceramic used. The rules for IPS e.max Press are described in the Instructions for Use.

Investing

All investment materials from Ivoclar Vivadent are suitable for the investment of these wax objects. Please observe the corresponding manufacturer's instructions when investing these or other press ceramic restorations.

Storage

- Protect from direct sunlight.
- Store out of the reach of children.
- Store at 2–28 °C / 36 – 82 °F.

Disposal

ProArt CAD Wax yellow is made from pure synthetic wax. Residues can be used for other customary wax applications. The wax can be disposed of in the general household waste.

This material has been developed solely for use in dentistry. Liability cannot be accepted for damages resulting from failure to observe the Instructions or the stipulated area of use. The user is responsible for testing the material for its suitability and use for any purpose not explicitly stated in the Instructions. These regulations also apply if the material is mixed or used in conjunction with products of other manufacturers.

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