

OptraStick

WITH THE CONVENIENT ADHESIVE TIP



OptraStick

- easy pick-up
- strong hold
- precise placement

OptraStick – the application instrument with the convenient adhesive tip


Restorations can easily and quickly be picked up, held and placed with OptraStick. Even if only slight pressure is applied, small objects and restorations adhere precisely to the adhesive tip of OptraStick, such as

- inlays, onlays, crowns or veneers of **IPS Empress** and **IPS e.max**
- veneers of **IPS d.SIGN** und **IPS InLine** or
- inlays, onlays, ¾-crowns of **Academy Gold** und **Harmony Medium**



OptraStick Refill / 50 pieces

Clinical case with IPS e.max Press

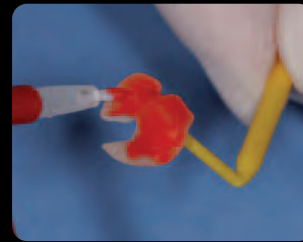

1. Application of IPS Ceramic Etching Gel




2. Removal of IPS Ceramic Etching Gel with water spray



3. Silanization with Monobond-S



4. Application of the luting composite Multilink Automix

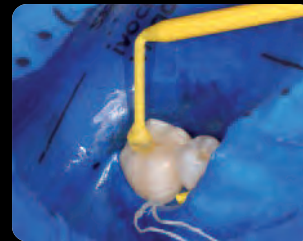

5. Insertion of the restoration



6. Removal of OptraStick using a hand instrument

Source: Dr R. Watzke, Ivoclar Vivadent AG, Schaan

ADDITIONAL USEFUL CLINICAL ACCESSORIES FOR INDIRECT RESTORATIONS



Anatomically shaped OptraDam® – for absolute isolation:

allows easy access to a significantly increased operation field.



OptraGate® ExtraSoft – for relative isolation:

provides gentle and even retraction of the lips and cheeks and gently creates a larger treatment field.



OptraFine® – highly efficient diamond polishing system:

highly aesthetic gloss can be achieved with the three-step polishing system for ceramic restorations. Suitable for intra- and extraoral application.