**Brugsanvisning  
Käyttöohjeet  
Bruksanvisning  
Productinformatie  
Οδηγίες Χρήσεως  
Kullanım kılavuzu  
Инструкция  
Instrukcja stosowania**

**Instructions for Use  
Gebrauchsinformation  
Mode d'emploi  
Istruzioni d'uso  
Instrucciones de uso  
Instruções de Uso  
Bruksanvisning**

- Mod dentinhyperéstesi
- Desensitointiaine
- Desensibiliseringsmiddel
- Desensibilisatiemiddel
- Απαισθητοποιητικός παράγοντας
- Hassasiyet giderici
- Десенситайзер
- Materiał do zmniejszania nadwrażliwości zębiny

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For dental use only.  
Caution: U.S. Federal law restricts this device to sale by or on the order of a licensed dentist.

Made in Liechtenstein  
Ivoclar Vivadent AG,  
FL-9494 Schaan/Liechtenstein

Complies with:  
ISO 4049, EN ISO 4049

**CE 0123**

**ivoclar**  
**vivadent**  
clinical

6285490610WEE3H

## Instructions for Use

### Description:

Telio CS Desensitizer is a material for reducing and preventing dental hypersensitivity and postoperative sensitivity. The material has been developed from the Syntac dentin adhesive system. Telio CS Desensitizer is suitable for the following fields of application: temporary restorations, luting/ cementation of indirect restorations, direct restorative treatment and treatment of sensitive tooth necks. Telio CS Desensitizer is a clear solution for application onto dentin (enamel may also be coated).

### Composition:

Telio CS Desensitizer contains polyethylene glycol dimethacrylate and glutaraldehyde in an aqueous solution.

### Indication:

For reducing and preventing dentinal hypersensitivity and postoperative sensitivity in the following cases:

- Temporary restorations
- Luting / cementation of indirect restorations
- Direct restorative treatment
- Sensitive tooth necks

### Contraindication:

- If a patient is known to be allergic or hypersensitive to any of the material's ingredients
- If the pain is caused by pulpitis or inflammation

### Side effects:

In rare cases, ingredients of Telio CS Desensitizer may cause a sensitizing reaction in patients with a hypersensitivity to any of the material's ingredients. In these cases, the material should no longer be used.

### Interactions:

Interaction with other materials (adhesives, cements, temporaries, restoratives) applied to the tooth structure may occur. These interactions may, for example, affect the bond strength of adhesives or cements.

## Application procedure:

### a) Short description of the application of

#### Telio CS Desensitizer

##### – Prior to temporization procedures, luting procedures, direct restorative procedures (amalgam, cement restorations):

- Dentin surfaces must be clean and dry. Keep working field dry (e.g. cotton rolls).
- Apply Telio CS Desensitizer on dentin and gently brush it into the dentin for 10 seconds using a suitable instrument (brush, applicator brush).
- Carefully disperse excess to a thin layer with blown air / dry with an air syringe. Do not overdry the dentin.
- Continue with the planned treatment measures (temporary restorative procedures / luting, cementation, direct restorative procedures).

##### – Sensitive tooth necks

- Dentin surfaces must be clean and dry. Keep working field dry (e.g. cotton rolls).
- Apply Telio CS Desensitizer and gently brush it into the dentin for 10 seconds using a suitable instrument (brush, applicator brush).
- Wait 20 seconds (improves penetration of Telio CS Desensitizer into the often sclerotic and less penetrable root dentin of sensitive tooth necks).
- Carefully disperse excess to a thin layer with blown air / dry with an air syringe. Do not overdry the dentin.
- Repeat application if the complaints recur or persist.

### b) Detailed instructions for the individual indications

#### 1. Temporary restorations

- Telio CS Desensitizer makes the temporary restoration phase more comfortable for patients, since it reduces the sensitivity of prepared, and in some cases exposed, dentin surfaces.
- Prepare cavity/abutment.
- You may take the impression at this stage or wait until you have applied the desensitizer (see notes).
- Dentin surfaces must be clean and dry. Keep working field dry (e.g. cotton rolls).

- Apply Telio CS Desensitizer and gently brush it into the dentin for 10 seconds using a suitable instrument (brush, applicator brush).
- Carefully disperse excess to a thin layer with blown air / dry with an air syringe. Do not overdry the dentin.
- Fabricate the temporary restoration / provisionally cement restoration

#### Notes:

- The application of Telio CS Desensitizer somewhat increases the adhesion of the light-curing temporary inlay and onlay materials from Ivoclar Vivadent to the cavity walls. Nevertheless, they can be removed at the second appointment without difficulty.
  - If light-curing temporary C&B materials are used, Telio CS Desensitizer should be applied after the fabrication of the temporary. If you apply Telio CS Desensitizer prior to fabricating the temporary, the material may stick to the tooth structure when the impression is repositioned into the oral cavity during the setting/light-curing procedure and may be difficult to remove.
  - The application of Telio CS Desensitizer during temporary fabrication does not affect or limit the choice of the cementation system or the success of the cementation procedure in the second appointment. Both conventional and adhesive cementation systems can be used for cementation purposes.
  - The application of Telio CS Desensitizer to enamel does not create any problems and does not affect the subsequent steps of the treatment or the materials being used (thoroughly remove excess desensitizer / dry)
  - Telio CS Desensitizer can be applied before or after impression-taking. The layer thickness is very thin, so that the accuracy of fit is maintained, even if the desensitizer is applied after impression-taking. Interaction of the desensitizer with impression materials are not known (if applied before impression-taking).
- #### 2. Luting / Cementation
- (Application in conjunction with conventional cements such as phosphate cements or glass ionomer cements)
- Remove temporary.

- Clean preparation, e.g. with a brush, rubber cup, and cleaning paste.
- Dentin surfaces must be clean and dry. Keep the working area dry (e.g. cotton rolls).
- Apply Telio CS Desensitizer and gently brush the material into the dentin for 10 seconds using a suitable instrument (brush, applicator brush).
- Carefully disperse excess to a thin layer with blown air / dry with an air syringe. Do not overdry the dentin.
- Try in restoration.
- Seat restoration with a suitable cementation material (phosphate cement, glass ionomer cement, carboxylate cement, reinforced glass ionomer cement).

#### Notes:

- Coating of enamel areas with Telio CS Desensitizer does not create any problems and does not affect the subsequent treatment steps and materials being used (thoroughly remove excess desensitizer / dry).

### 3. Direct restorations

#### a) Application in conjunction with amalgam, glass-ionomer cement restorative materials, etc

- Prepare cavity, excavate caries.
- Dentin surfaces must be dry and clean. Keep working area dry (rubber dam, e.g. OpraDam® Plus)
- Apply Telio CS Desensitizer and gently brush the material into the dentin for 10 seconds using a suitable instrument (brush, applicator brush). (Coating of enamel with Telio CS Desensitizer does not create any problems and does not affect the subsequent treatment steps and materials being used).
- Carefully disperse excess to a thin layer with blown air / dry with an air syringe. Do not overdry the dentin.
- Apply a base / liner, if necessary.
- Fabricate restoration with a suitable restorative material.

- b) Application in conjunction with dentin adhesives: Telio CS Desensitizer may also be used in conjunction with dentin adhesives. In this case, Telio CS Desensitizer acts as a rewetting agent that restores the appropriate moisture contents of the dentin after drying or even overdrying.**

- Apply Telio CS Desensitizer by brushing it into the dentin for 10 seconds
- Disperse excess with a slow stream of blown air
- Dry using the air gun
- Apply the adhesive according to the instructions of the manufacturer

#### Warning:

Telio CS Desensitizer contains glutaraldehyde and is hazardous to health if swallowed or inhaled. Avoid contact with the skin, mucous membrane and the eyes. If the material comes into contact with the eyes, immediately rinse with copious amounts of water and seek medical advice. Repeated contact of the material with the skin or inhalation may result in sensitization. Commercially available medical gloves do not provide protection against the sensitizing effect of methacrylates.

#### Storage:

- Shelf life: see date of expiration on label/package
- Do not use Telio CS Desensitizer after the date of expiration.
- Immediately close the bottle after use.
- Storage temperature: 2-28°C (36 – 82°F).

#### Keep out of the reach of children!

#### For use in dentistry only!

Date information prepared: 06/2010; Rev. 1

#### Manufacturer

Ivoclar Vivadent AG,  
FL-9494 Schaan/Liechtenstein

This material has been developed solely for use in dentistry. Processing should be carried out strictly according to the Instructions for Use. Liability cannot be accepted for damages resulting from failure to observe the Instructions or the stipulated area of application. The user is responsible for testing the materials for their suitability and use for any purpose not explicitly stated in the Instructions. Descriptions and data constitute no warranty of attributes and are not binding.

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## Ivoclar Vivadent UK Limited
## Ivoclar Vivadent, Inc.