





BASIC infiltration with IPS e.max® ZirCAD MT/LT Colouring Liquids A–D

- 25  IPS e.max® ZirCAD MT/LT Colouring Liquids (A–D)
- + 1  IPS e.max® ZirCAD Colouring Liquid Indicator red



1


1x with brush No. 5



2




1x with brush No. 5

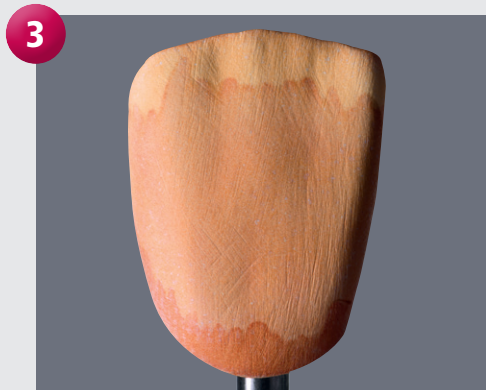


3

1x with brush No. 5

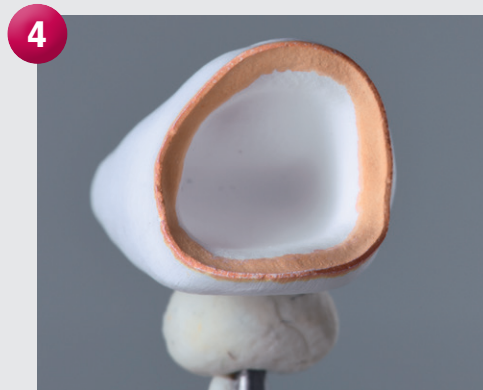


4

1x with brush No. 5

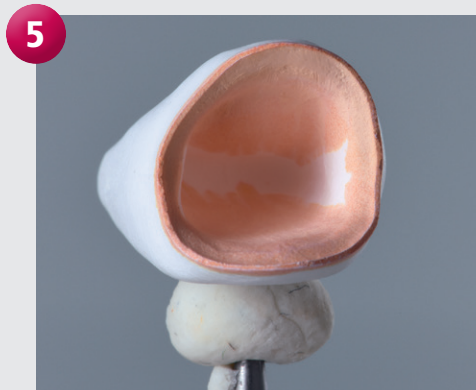

5

1x with brush No. 5

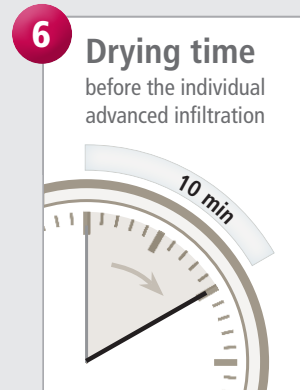

6

Drying time
before the individual
advanced infiltration



Brush No. 1

Brush No. 3



Brush No. 5





ADVANCED infiltration with IPS e.max® ZirCAD MT/LT Colouring Liquids A–D

Recommendations



for shading the Effect shades with IPS e.max® ZirCAD Colouring Liquid Indicator:

- 25  IPS e.max® ZirCAD MT/LT Colouring Liquid grey, violet
- +
- 1  IPS e.max® ZirCAD Colouring Liquid Indicator yellow



- 25  IPS e.max® ZirCAD MT/LT Colouring Liquid orange
- +
- 1  IPS e.max® ZirCAD Colouring Liquid Indicator blue



- 25  IPS e.max® ZirCAD MT/LT Colouring Liquid blue
- +
- 1  IPS e.max® ZirCAD Colouring Liquid Indicator blue + yellow (= green)



Apply grey in the incisal and marginal ridge area for a lifelike in-depth effect

1x with brush No. 1 or 3



Translucent effects with blue in the incisal area

1x with brush No. 1 or 3



Translucent effect with violet in the incisal and cervical area

1x with brush No. 1 or 3



10 Drying time before sintering

