# OptraFine

## Instructions for use Gebrauchsinformation

## Mode d'emploi Istruzioni d'uso Instrucciones de uso Instruções de Uso Brugsanvisning Brugsanvisning

- Diamond polishing system for ceramic materials
- Diamant-Poliersystem für Keramikmaterialien
- Système de polissage pour matériaux céramique
- Sistema di lucidatura diamantato per manufatti ceramici
- Sistema de pulido diamantado para materiales cerámicos
- Sistema diamantado para polimento de materiais cerâmicos
- Diamant-polersystem för keramiska material
- Diamant-poleringsssystem til keramiske materialer

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### Bruksveiledning

### Productinformatie

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## Instructions for use

#### Description

OptraFine is a 3-step polishing system with diamond particles suitable for polishing dental ceramic materials to a high lustre during clinical procedures. OptraFine consists of the following four components:

- Light-blue diamond finishers (F) in flame, cup and disc shapes
- Dark-blue diamond polishers (P) in flame, cup and disc shapes
- Nylon brushes for high-gloss polishing (HP), suitable for use in conjunction with the
- Diamond polishing paste for high-gloss polishing (HP)

#### Composition

- The finishers and polishers contain synthetic rubber, diamond granulate and titanium dioxide. The shanks are made of stainless steel.
- The nylon brushes consist of nylon fibres. The shanks are also made of stainless steel.
- The diamond polishing paste contains diamond dust with a particle size ranging from 2 to 4 µm in an emulsion of glycerine, sodium lauryl sulphate and propylene glycol.

#### Indications

High-gloss polishing of dental ceramic restorations

#### Contraindications

The products should not be used if a patient is known to be allergic to any of the ingredients of the polishing instruments or polishing paste.

#### Application

First, the surface of the ceramic restoration is finished and smoothed with the OptraFine F finishers under water spray. The subsequent polishing procedure is performed with the OptraFine P polishers also with water spray. For final high-lustre polishing, the OptraFine HP nylon brushes are used together with the diamond polishing paste without water spray. Prior to polishing, diamond polishing paste is dispensed into the cavity created by the circle of bristles. Subsequently, the paste is spread onto the restoration surface with the brush. Any polish or grinding residue can be removed with water spray.

#### Diamond finishers and polishers:

- Rotary speed: 10,000 up to max. 15,000 rpm
- Only use **with copious water spray** (> 50ml/min)
- Use without paste
- Immerse the finishers and polishers in an instrument cleaning bath for max. 1 hour
- Sterilize the finishers and polishers in the autoclave at 134 °C/275 °F

#### Brushes/polishing paste:

- Rotary speed: 7,000 up to max. 10,000 rpm
- Use **without water spray**
- Immerse the brushes in an instrument cleaning bath for max. 1 hour
- Sterilize brushes in the autoclave at 134 °C/275 °F

#### Rotary speed

- The recommended and maximum admissible rotary speeds indicated in the Instructions for Use and on the packaging must be observed. They must not be exceeded.
- The recommended and maximum admissible rotary speed is dependent on the polishers or polishing system used and may differ from system to system.
- The polishing instruments tend to vibrate if the maximum admissible rotary speed is exceeded. This may lead to the destruction of the polishing instrument, deformation of the shank and/or fracture of the instrument and thus poses a potential danger to the user/practice team and the patient.

#### Warning

**Failure to observe the maximum admissible rotary speed results in a heightened safety risk.**

#### Proper use

- Use only turbines and straight/contra-angle handpieces which are technically and hygienically impeccable and regularly serviced and cleaned.
- The handpiece and turbine drives must run smoothly.
- The instruments must be correctly inserted and secured.
- Polishing should be performed with a slight rotary motion to avoid the creation of grooves
- Jamming/leveraging should be avoided to reduce the risk of instrument fracture.

- Deformed instruments that do not run smoothly should cease to be used.
- From the point of view of safety, protective goggles and breathing protection should always be worn in order to avoid inhaling grinding dust. Efficient dust evacuation must be ensured during the polishing procedure.

#### Warning

**Improper use may lead to poor polishing results and a heightened health risk. This product must only be used by skilled persons.**

#### Contact pressure

High contact pressure during polishing will not produce an improved polishing result, but will lead to the increased generation of heat and accelerated wear of the polishing instrument. A mean contact pressure of approx. 2 N is sufficient.

#### Warning

**When polishing intraorally, excessive pressure needs to be avoided in order to prevent overheating and eventually pulp damage. In extreme cases, instrument fracture may result if too much pressure is applied, which can lead to injuries.**

#### Water cooling

In order to avoid unwanted heat generation on the tooth surface, sufficient water cooling (at least 50 ml/min) needs to be ensured. In addition, any residue produced during polishing is rinsed off by the water spray.

#### Warning

**Insufficient water cooling may lead to irreversible damage of the tooth and the adjacent tissues.**

#### Storage, cleaning, disinfection and sterilization

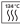
- The polishing instruments should be stored in the original packaging at room temperature and protected from dust, moisture, pressure and contamination.
- Before using the polishing instruments on the patient for the first time, they must be disinfected or sterilized.
- After having used the polishing instruments in the mouth of the patient, they must be cleaned immediately and disinfected or sterilized.
- When manipulating contaminated instruments, protective clothing and gloves should be worn.
- Care needs to be taken that the selected disinfectant is suitable for the disinfection of finishers/polishers/nylon brushes and that the instruments are disinfected correctly according to the instructions of the manufacturer. Please refer to the instructions of the manufacturer for details on recommended use, immersion time and suitability of disinfection and cleaning agents. Aggressive substances, excessively long immersion times or incorrect dosage may damage the instruments.
- Sterilization must be done in the vacuum autoclave for 20 minutes at 134 °C or according to RKI guidelines. A temperature of 150 °C should not be exceeded.
- The products are not suitable for sterilization in the chemiclave or steam disinfectator.

#### Warning

**The suitability of the selected method for the disinfection/ sterilization of the respective instrument must be verified in advance.**

#### Storage and shelf life

- Storage temperature 2 - 28 °C/36 - 82 °F
- Expiry date: See imprint on label
- Immediately seal polishing paste after use.

-  Sterilizable in the autoclave at the temperature indicated
- The respective instructions for use can be downloaded from the Ivoclar Vivadent website.

#### Keep out of the reach of children!

#### For use in dentistry only!

**Date information prepared:** 09/2010 Rev. 1

#### Manufacturer:

Ivoclar Vivadent AG, FL-9494 Schaan/Liechtenstein

The product has been developed solely for use in dentistry. Processing should be carried out strictly according to the Instruc-tions for Use. Liability cannot be accepted for damages resulting from failure to observe the Instructions or the stipulated area of application. The user is responsible for testing the products for their suitability and use for any purpose not explicitly stated in the Instructions. Descriptions and data constitute no warranty of attributes

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#### Ivoclar Vivadent Limited
#### Ivoclar Vivadent, Inc.