EN Instructions for Use

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Description

Monobond® Plus is a universal primer promoting an adhesive bond between luting composites (particularly the Variolink and Multilink product lines) and all indirect restorative materials (glass and oxide ceramics, metal, composites, fiber-reinforced composites).

Composition

Alcohol solution of silane methacrylate, phosphoric acid methacrylate and sulphide methacrylate.

Indication

Monobond Plus serves as bonding agent and is used to create a durable chemical bond between luting composites and glass/oxide ceramic, metal, composite and fiber-reinforced composite restorations.

Contraindications

- Any use not listed in the indications.
- If a patient is known to be allergic to any of the ingredients of Monobond Plus.

Side effects

To date, no systemic side effects are known.

In rare cases, components of Monobond Plus may lead to sensitization. Monobond Plus should not be used in such cases.

Interactions

Contamination with saliva, blood or water must absolutely be avoided during the entire procedure.

Application

Adhesive cementation of prosthetic restorations:

- 1 Preparing the restoration
 - 1.1 Glass-ceramic restorations (e.g. IPS Empress®)
 - Etch with 5% hydrofluoric acid (e.g. IPS® Ceramic Etching Gel) for 60 seconds or according to the instructions of the manufacturer of the restorative material.
 - Thoroughly rinse with water spray and dry with oil-free air.
 - 1.2 Lithium disilicate glass-ceramic restorations (e.g. IPS e.max® Press, IPS e.max CAD)

Rx ONLY

Date information prepared:

2013-07-16/Rev. 4
626277/WE3

CE 0123

Manufacturer
Ivoclar Vivadent AG
FL-9494 Schaan/Liechtenstein
www.ivoclarvivadent.com

ivoclar
vivadent®
clinical

- Etch with 5% hydrofluoric acid (e.g. IPS® Ceramic Etching Gel) for 20 seconds or according to the instructions of the manufacturer of the restorative material.
- Thoroughly rinse with water spray and dry with oil-free air.
- 1.3 Zirconium oxide (e.g. IPS e.max ZirCAD) or aluminium oxide ceramics
 - Sandblast the restoration surface (sandblasting parameters according to the instructions of the manufacturer of the restorative material).
 - If necessary, clean the restoration in an ultrasonic unit for about 1 minute.
 - Thoroughly rinse with water spray and dry with oil-free air.
 - **IMPORTANT!** In order to create a strong bond, do not clean the oxide ceramic surfaces with phosphoric acid.
- 1.4 Metal or metal-supported restorations
 - Sandblast the restoration surface (sandblasting parameters according to the instructions of the manufacturer of the restorative material) until an even, matt or non-glossy surface has been achieved.
 - If necessary, clean the restoration in an ultrasonic unit for about 1 minute.
 - Thoroughly rinse with water spray and dry with oil-free air.
 - **IMPORTANT!** In order to create a strong bond, do not clean the metal surfaces with phosphoric acid.
- 1.5 Composite or fiber-reinforced composite restorations
 - Sandblast the restoration surface (according to the instructions of the manufacturer of the restorative material).
 - If necessary, clean the restoration in an ultrasonic unit for about 1 minute.
 - Thoroughly rinse with water spray and dry with oil-free air.
- 2 Apply a **thin** coat of Monobond Plus with a brush or a Microbrush to the pre-treated surfaces (avoid pooling when treating crowns). Allow the material to react for 60 seconds. Subsequently, disperse any remaining excess with a strong stream of air.
- 3 Apply the luting composite (e.g. Variolink® II, Variolink® Veneer or MultiLink® Automix) according to the respective Instructions for Use and insert the restoration.

Special notes

- Contamination with saliva, blood or water must absolutely be avoided during the entire procedure. In case of contamination, the entire treatment of the restoration is to be repeated (see section 1).
- **IMPORTANT!** In order to create a strong bond, do not clean the zirconium oxide surfaces with phosphoric acid.
- Do not insert the brush into the Monobond Plus bottle. This may lead to contamination.
- Carefully close the Monobond Plus bottle immediately after use to prevent evaporation of the solvent.

Warning

- The product contains ethanol. Keep away from ignition sources.
- Monobond Plus is an irritant. Avoid contact with the skin and mucous membrane. If the material should accidentally come in contact with the skin, immediately wash with copious amounts of water.
- In rare cases, contact with the skin may lead to sensitization towards the ingredients.
- Commercial medical gloves do not provide protection against the sensitizing effect of methacrylates.

Storage

- Monobond Plus should never be used after the expiration date.
- Storage temperature: 2–28 °C / 36–82 °F.
- Shelf life: see expiry date on label.

Keep out of the reach of children.

For use in dentistry only.

The material has been developed solely for use in dentistry. Processing should be carried out strictly according to the Instructions for Use. Liability cannot be accepted for damages resulting from failure to observe the Instructions or the stipulated area of application. The user is responsible for testing the material for its suitability and use for any purpose not explicitly stated in the Instructions. Descriptions and data constitute no warranty of attributes and are not binding.

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Ivoclar Vivadent Limited
Ivoclar Vivadent, Inc.