### Instructions for Use Gebrauchsinformation

#### Mode d'emploi

#### Istruzioni d'uso

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#### Bruksanvisning

#### Bruksanvisning

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#### Bruksanvisning

#### Productinformatie

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For dental use only.

Caution: U.S. Federal law restricts this device to sale by or on the order of a licensed dentist.

Made in Liechtenstein  
Ivoclar Vivadent AG,  
FL-9494 Schaan/Liechtenstein

  
ivoclar  
vivadent®  
clinical

### Description

Ivoclean is a universal cleaning paste suitable for non-abrasive cleaning of the bonding surfaces of prosthetic restorations after intraoral try-in. Consequently, it helps to achieve reliable adhesive cementation results.

### Composition

Dispersion of metal oxide particles in water.

### Indication

Extraoral cleaning of pre-treated ceramic and metal restoration surfaces which have been contaminated during intraoral try-in.

### Contraindication

Ivoclean is contraindicated:

- if the stipulated working technique cannot be applied;
- if cleaning is performed intraorally;
- for the cleaning of restorations which have come in contact with silicone-based pastes.

### Interactions

After cleaning, the bonding agent must be reapplied to surfaces that were previously silanized and primed.

### Application

Shake bottle before use.

- Independent of whether or not the restorations have been conditioned beforehand, they should be cleaned according to the following cleaning protocol:
- After try-in, thoroughly rinse the restoration with water spray and dry with oil-free air.
  - Cover the entire bonding surface of the restoration with a layer of Ivoclean using a microbrush or bush.
  - Allow 20 seconds for the cleaning action of Ivoclean to take effect, then thoroughly rinse with water spray and dry with oil-free air.
  - Next, prime the bonding surface of the restoration with a suitable bonding agent (e.g. Monobond Plus). Make sure to observe the instructions for use of the bonding agent and the luting material used.

### Additional notes

Contamination of the restoration with saliva or blood must be absolutely avoided after cleaning with Ivoclean. In case of contamination, the entire treatment of the restoration must be repeated. After cleaning, the bonding agent must be reapplied to surfaces that were previously silanized and primed.

### Warning

- Causes burns.
- In case of accident or if you feel unwell, seek medical advice immediately.
- Do not breathe fumes/aerosols.

- In case of contact with eyes, rinse immediately with plenty of water and seek medical advice.
- Wear suitable protective clothing, gloves and eye/face protection.
- When using do not eat or drink.
- This material and its container must be disposed of as hazardous waste.

## Storage

- Store Ivoclean at 2-28°C (26-82 °F). Storage stability can only be guaranteed, if the product is stored at the temperature indicated.
- Close bottle immediately after use.
- Shelf life: see expiry date on label.

## Keep out of the reach of children!

### For use in dentistry only.

## Date information prepared:

04/2011, Rev 0

## Manufacturer:

Ivoclar Vivadent AG  
FL-9494 Schaan/Liechtenstein

The material has been developed solely for use in dentistry. Processing should be carried out strictly according to the Instructions for Use. Liability cannot be accepted for damages resulting from failure to observe the Instructions or the stipulated area of application. The user is responsible for testing the products for their suitability and use for any purpose not explicitly stated in the Instructions.

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## Ivoclar Vivadent Limited
## Ivoclar Vivadent, Inc.