

Programat® P310

Software Revision Log

Version	Changes from previous version	Status
V5.2 11.03.2019	Software optimized <ul style="list-style-type: none"> • New program for IPS Style paste opaquer • Optimized vacuum test program • Optimized dehumidification-function • More special characters • Acoustic feedback of touchscreen • Various languages have been improved • Various software optimizations 	Currently delivered version
V5.1 01.02.2018	Software optimized <ul style="list-style-type: none"> • Push function can be tested • A copy wizard is now available • Various languages have been improved • Various software optimizations 	No longer available!
V5.0 28.04.2017	Software optimized <ul style="list-style-type: none"> • New program structure with new program parameters for IPS Style, IPS e.max Press Layering Technique, IPS e.max ZirPress Layering Technique, IPS e.max ZirCAD Staining Technique, IPS e.max CAD Crystall/Glaze, IPS e.max CAD Layering Technique, IPS e.max CAD Layering Technique, IPS d.SIGN, IPS e.max ZirCAD Layering Technique • Optimized Vacuum test program • Various software optimizations 	No longer available!
V4.0 08.04.2016	Software optimized <ul style="list-style-type: none"> • New programs for IPS Style and IPS Ivocolor • In-process dehumidification: Function that protects the vacuum system from contamination and its consequential damages • Various software optimizations 	No longer available!
V3.1 09.04.2015	Software optimized <ul style="list-style-type: none"> • Various software optimizations 	No longer available!
V3.0 01.10.2014	Software optimized <ul style="list-style-type: none"> • New firing programs for IPS InLine Powder Opaquer (P21, P221, P241) • Various software optimizations 	No longer available!

Programat® P310

V2.1 06.06.2014	Software optimized <ul style="list-style-type: none"> The software was attuned to meet the requirements of the Programat App (for iPhones) and various software optimization measures were implemented 	No longer available!
V2.0 18.11.2013	Software optimized <ul style="list-style-type: none"> New Speed Crystallization program for IPS e.max CAD (Important: The speed program must only be used in combination with the specially designed IPS e.max CAD Speed Crystallization Tray) Additional software improvements 	No longer available!
V1.1 20.09.2013	Basic version	No longer available!

Version 11.03.2019