Cervitec® F

EN Instructions for Use

- Protective dental varnish

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- Verniz protetor dental

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TR Kullanma Talimatı

- Koruyucu dental vernik

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- Barniz dental protector

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English

Description

Cervitec® F is a dental varnish for the mechanical protection of the tooth structure. It delivers fluoride and has antimicrobial action. The varnish is intended for the professional use in the dental practice and for the use of professionally instructed personnel under field conditions. The innovative varnish system is characterized by its good moisture tolerance during application.

Composition

Alcohol, Aqua, VA/Crotonates Copolymer, Cetylpyridinium chloride, Chlorhexidine diacetate, Ammonium fluoride, Aroma, Saccharin

Contains ammonium fluoride. Fluoride content: 1400 ppm

Application

- Caries prevention in risk groups
- Fluoridation of dental enamel
- Remineralization of incipient caries
- Protection of hyper-sensitive teeth
- Antimicrobial action

If users are known to be allergic to any of the ingredients in Cervitec F, the material should not be applied.

Cervitec F is applied by the dentist or professionally instructed personnel. Cervitec F is suitable for all age groups.

Generally, Cervitec F is applied every three months. If required, it can be applied at shorter intervals.

Bottle 4 g: The contents are sufficient for about 17 full set of teeth.

Bottle 7 g: The contents are sufficient for about 30 full set of teeth.

Step-by-step description of the procedure

1. Clean the tooth surfaces.
2. Isolate the application area with cotton rolls and cotton swabs. The use of a saliva ejector or air syringe is optional.
3. Dispense the required amount into e.g. a dappen dish and close the bottle.
4. Apply the varnish in a thin layer using an applicator brush or brush.
5. Allow the varnish to dry for 1 minute and then remove the cotton rolls.
6. No rinsing after application, only spitting.

Eating and drinking should be avoided for 1 hour after application of Cervitec F.

Notes

Contact with the mucous membrane may result in short-term, reversible irritation. If the varnish comes into contact with the eyes, immediately rinse with copious amounts of water and consult an ophthalmologist.

Self life and storage

- Carefully close the bottle immediately after use.
- Highly flammable. Keep away from sources of ignition.
- Do not use after the indicated date of expiration.
- Store at 2–28 °C / 36–82 °F.

Keep out of the reach of children!

For use in dentistry only.

The material has been developed solely for use in dentistry. Processing should be carried out strictly according to the Instructions for Use. Liability cannot be accepted for damages resulting from failure to observe the Instructions of the stipulated area of application. The user is responsible for testing the products for their suitability and use for any purpose not explicitly stated in the Instructions. Descriptions and data constitute no warranty of attributes and are not binding.

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